

DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION **FACILITIES DEVELOPMENT DIVISION**

OFFICE USE ONLY APPLICATION FOR HCAI PREAPPROVAL OF **MANUFACTURER'S CERTIFICATION (OPM)** APPLICATION #: OPM-0082 **HCAI Preapproval of Manufacturer's Certification (OPM)** Renewal/Update New Type: **Manufacturer Information** Manufacturer: Steris Corporation Manufacturer's Technical Representative: Wael Abdul-Hadi Mailing Address: 490 Boul. Armand-Paris, Quebec, QC G1C8A3 Telephone: (418) 664-1549 Email: Wael Absul-Hadi@steris.com **Product Information** Product Name: AMSCO 2532 and RELIANCE 300 XLS Single-Chamber Washer/Disinfector Product Type: Other Electrical & Mechanical Elements Product Model Number: 2532 & 300 XLS General Description: Washer **Applicant Information** Applicant Company Name: EASE LLC. Contact Person: Tiffany Tonn Mailing Address: 1515 FAIRVIEW AVE, STE 205, MISSOULA, MT 59801

Email: tiffany@easeco.com

"A healthier California where all receive equitable, affordable, and quality health care"

STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY

Telephone: (406) 541-3273

Title: Office Manager



DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION FACILITIES DEVELOPMENT DIVISION

Registered Design Professonal Preparing Engineering Recommendations
Company Name: EASE LLC
Name: Jonathan Roberson California License Number: S4197
Mailing Address: 5877 Pine Ave., Suite 210, Chino Hills, CA 91709
Telephone: (951) 295-1892
HCAI Special Seismic Certification Preapproval (OSP)
Special Seismic Certification is preapproved under OSP OSP Number:
OR CODE C
Certification Method
Testing in accordance with: ICC-ES AC156 FM 1950-16
Other(s) (Please Specify): OPM-0082
*Use of criteria other than those adopted by the California Building Standards Code, 2022 (CBSC 2022) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2022 may be used when approved by HCAI prior to testing.
X Analysis
Experience Data DATE: 10/24/2023
Combination of Testing, Analysis, and/or Experience Data (Please Specify):
CPVIA DE CODE
HCAI Approval
Date: 10/24/2023
Name: William Staehlin Title: Senior Structural Engineer
Condition of Approval (if applicable):

"A healthier California where all receive equitable, affordable, and quality health care"

HCAi

STATE OF CALIFORNIA - HEALTH AND HUMAN SERVICES AGENCY



5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

The Department of Health Care Access and Information
PREAPPROVAL OF MANUFACTURER'S CERTIFICATION

OPM-0082

THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE

MANUFACTURER:

STERIS CORPORATION

Sheet: 1 of 8

EQUIPMENT NAME:

AMSCO 2532 SINGLE CHAMBER WASHER/DISINFECTOR

Date: 10/4/23


AND RELIANCE 300XLS GLASSWARE WASHER

GENERAL NOTES

- 1. THIS HCAI PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2022 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2022 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE WHERE SDS IS NOT GREATER THAN 1.50 & 2.30. SEE DETAIL FOR APPLICABILITY
- 4. FORCES PER ASCE 7-16 SECTION 13.3.1, EQUATIONS 13.3-1, 13.3-2 & 13.3-3, WHERE SDS = 1.50, a_p = 1.0, I_p = 1.5, R_p = 1.5, z/h = 0 AT CONCRETE SLAB. SEE FOLLOWING SHEETS FOR Ω_o WHERE SDS = 2.30, a_p = 1.0, I_p = 1.5, R_p = 1.0, z/h = 0 AT CONCRETE SLAB & $z/h \le 1$ AT CONCRETE SLAB ON METAL DECK. SEE FOLLOWING SHEETS FOR Ω_o
- 5. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- 6. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. CONCRETE SLAB ON METAL DECK DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING. (i.e. z/h < 1)
- 8. CONCRETE SLAB DETAIL VALID FOR DEMANDS SHOWN AT OR BELOW GRADE. (i.e. z/h = 0)

9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE TO SUPPORT WEIGHTS AND FORCES SHOWN IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2022 CBC AND WITH THE DETAILS, MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT PROJECT SPECIFIC VALUES OF SDS & z/h RESULT IN SEISMIC FORCES (Eh, Ev) THAT DO NOT EXCEED THE VALUES ON THE DETAILS.
- D. VERIFY THAT THE CONCRETE SLAB TO WHICH THE EQUIPMENT IS ANCHORED MEETS THE REQUIREMENTS OF THE APPLICABLE ICC ESR REPORT. AND THIS OPM.
- E. VERIFY THAT THE ANCHORS ARE AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPENINGS (SEE TYPICAL DETAIL ON SHEET 2).
- F. VERIFY THAT ALL NEW OR EXISTING ANCHORS ARE AN ADEQUATE DISTANCE FROM THE UNIT ATTACHMENTS AND CHECK FOR INTERACTION WHERE OTHER ANCHORS ARE WITHIN 18" OR 6hef FROM THIS UNIT'S ANCHORS.

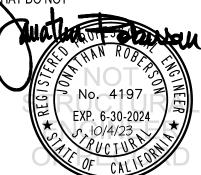

www.EquipmentAnchorage.com

STERIS CORPORATION

DES. J. ROBERSON JOB NO. 14-2307

SHEET 2

AMSCO-2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER

DATE 10/4/23 OF 8 SHEETS

10. EXPANSION ANCHORS:

A. ATTACHMENT IS TO BE MADE WITH THE ANCHORS LISTED BELOW AND INSTALLED AS DESCRIBED IN THE CORRESPONDING ICC REPORT.


Anchor Diameter	Concrete Type	Min. f'c (psi)	Anchor Type	ICC Report No.	Min. Embed.	Min. Spacing	Min. Edge Dist.	Min. Conc. Thickness	Torque Test	Direct Tension Test
3/8"	Sand Light Weight	3000	Hilti Kwik Bolt TZ2	ESR-4266	2"	6.75"	12"	See Detail "A"	30 FT-LB	N/A
3/8"	Normal Weight	3000	Hilti Kwik Bolt TZ2	ESR-4266	2"	8"	14"	4	30 FT-LB	1983 lb
3/8"	Normal Weight	3000	Hilti HIT-HY 200 V3	ESR-4868	6.5	12"	12"	8"	N/A	4370 lb

- B. THIS PREAPPROVAL ALLOWS FOR UP TO A MAXIMUM OF 2 ADJACENT CONCRETE SLAB EDGES, 12" & 14 AWAY MINIMUM (i.e. - CORNER).
 SEE ADJACENT DETAIL FOR ADDITIONAL MINIMUM ALLOWABLE CONCRETE
 EDGE DISTANCES.
- C. TESTING AND SPECIAL INSPECTION OF EXPANSION ANCHORS SHALL
 BE PERFORMED BY AN APPROVED INDEPENDENT AGENCY
 EMPLOYED BY THE FACILITY OWNER PER CBC 1704A & 1910A.5

 AND CAC 7-149. ALL REPORTS SHALL BE SENT TO THE INSPECTOR
 OF RECORD, OWNER AND THE ARCHITECT OR ENGINEER IN I AM Staehlin
 RESPONSIBLE CHARGE.
 - (i) AFTER AT LEAST 24 HOURS HAVE ELAPSED SINCE INSTALLATION, DIRECT PULL TENSION TEST OR TORQUE TEST AT LEAST 50% OF 4/2023 THE ANCHORS.
 - (ii) ACCEPTANCE CRITERIA:
 - DIRECT TENSION TEST: THE ANCHOR SHOULD HAVE NO OBSERVABLE MOVEMENT AT THE TEST LOAD. A PRACTICAL WAY TO DETERMINE OBSERVABLE MOVEMENT IS THAT THE WASHER BECOMES LOOSE.
 - TORQUE TEST: THE APPLICABLE TORQUE MUST BE ACHIEVED WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: 1/2 TURN OF THE NUT
 - (iii) IF ANY ANCHOR FAILS, TEST ALL ANCHORS.
- D. AVOID DAMAGING EXISTING STEEL REINFORCING IN CONCRETE SLAB WHEN INSTALLING CONCRETE EXPANSION ANCHORS.
- E. PROVIDE FOR FULL THREAD ENGAGEMENT OF NUT & WASHER.

11. BOLTS THROUGH CONCRETE ON METAL DECK

- A. BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUTS AFTER THE SNUG TIGHT (THE SNUG-TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQUIRED TO BRING THE CONNECTED PLIES INTO FIRM CONTACT) CONDITION IS ACHIEVED. UNLESS OTHERWISE NOTED.
- B. THROUGH BOLT HOLES SHALL BE 1/16" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE + 1/16) FOR CONCRETE.
- C. THROUGH-BOLTS IN CONCRETE SHALL RECEIVE SPECIAL INSPECTION AND TESTING (THROUGH BOLTS WITH STEEL TO STEEL CONNECTION IN TENSION DO NOT REQUIRE TENSION TESTING) IN ACCORDANCE WITH REQUIREMENTS FOR POST-INSTALLED ANCHORS.

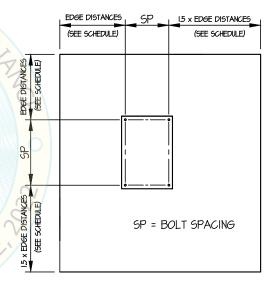

TYPICAL CONCRETE EDGE DETAIL (SLAB ON GRADE ONLY)



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER


DES. J. ROBERSON

JOB NO. 14-2307

DATE 10/4/23

3

F 8 SHEETS

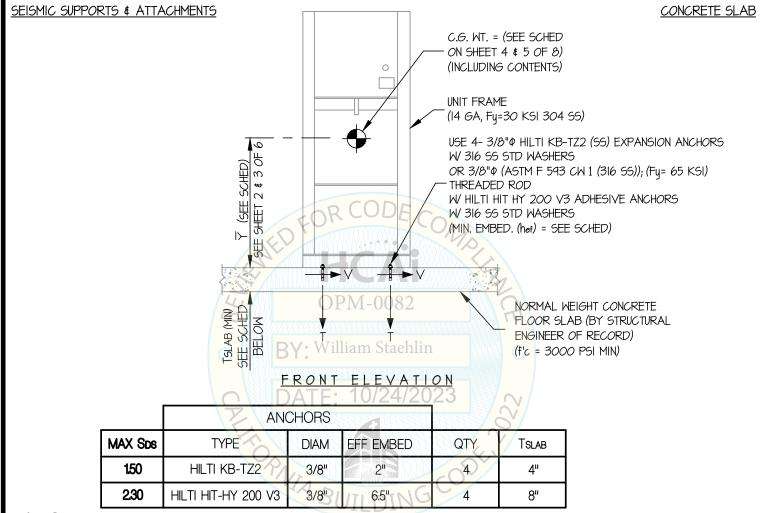

NOTES:

- 1. FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: 2p = 1.0, p = 1.5, p = 1.5, p = 1.5, p = 2.0, p = 2.0,
- 2. THIS PREAPPROVAL ENCOMPASSES WEIGHTS AND VERTICAL CG POSITIONS UP TO THE VALUES SHOWN.
- 3. THIS CALCULATION WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION. COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR). USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS, AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 4. SEE GENERAL NOTES: SHEETS 1 AND 2



www.EquipmentAnchorage.com

STERIS CORPORATION

DES. J. ROBERSON

14-2307

4

SHEET


AMSCO 2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER

DATE 10/4/23

JOB NO.

OF 8 SHEETS

CONCRETE SLAB

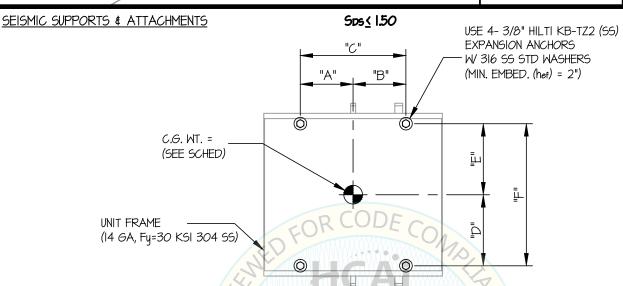

PLAN AT BASE

UNIT	WEIGHT (lb.)	"Y" (in.)	"A" (in.)	"B" (in.)	"C" (in.)	"D" (in.)	"E" (in.)	"F" (in.)	** Tu (lb.)	** Vu (lb.)
RELIANCE 300 XLS	646DA	35.88	8,88.4	/28.122	3 17	11.25	12.25	23.5	1063	296
AMSCO 2532 SINGLE CHAMBER	1054	31.3	8.5	8.5	17	11.75	11.75	23.5	1436	462

^{**} VALUES INCLUDE $\Omega_{\!\scriptscriptstyle 0}$



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER

DES. J. ROBERSON

JOB NO. 14-2307

DATE 10/4/23


SHEET 5

of 8 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS


Sps ≤ 2.30

CONCRETE SLAB

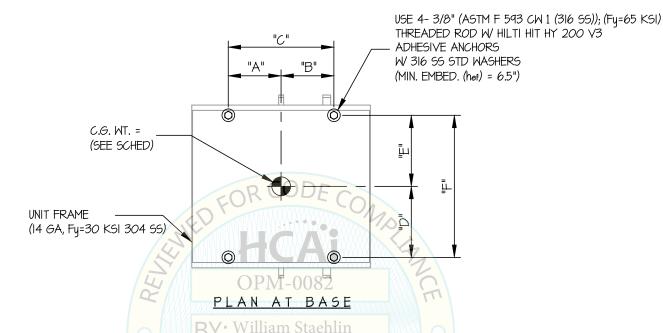

UNIT	WEIGHT (lb.)	"Y" (in.)	"A" (in.)	"B" (in.)	"C" (in.)	"D" (in.)	"E" (in.)	"F" (in.)	** Tu (lb.)	** Vu (lb.)
RELIANCE 300 XLS	646 DA	35.88	8.88	8.12	3 17	11.25	12.25	23.5	1714	454
AMSCO 2532 SINGLE CHAMBER	1054	31.3	8.5	8.5	17	11.75	11.75	23.5	2328	709

^{**} VALUES INCLUDE $\Omega_{\rm o}$

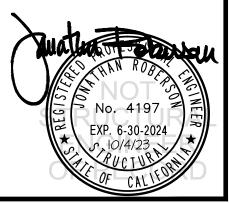

www.EquipmentAnchorage.com

STERIS CORPORATION

AND RELIANCE 300XLS GLASSWARE WASHER

DES. J. ROBERSON 14-2307 JOB NO.


SHEET

AMSCO-2532 SINGLE CHAMBER WASHER/DISINFECTOR 10/4/23 DATE

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

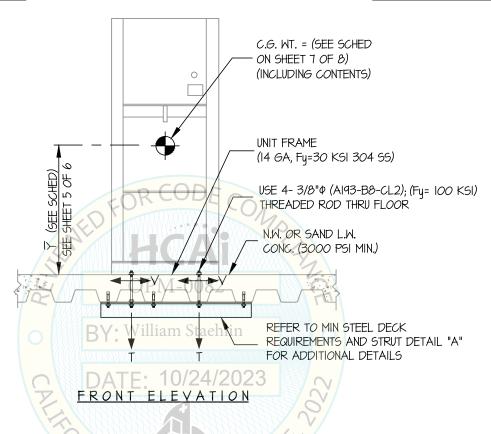

NOTES:

1. FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: SDS = 2.30, 2p = 1.5, 2p = 1.5, 2p = 1.5, 2p = 2.0, 2p = 2.0, 2p = 1.5, 2p =

> = 2.76 WpHORIZONTAL FORCE (En)

HORIZONTAL FORCE (Emh) = 5.52 Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (Ev) = 0.46 Wp

- 2. THIS PREAPPROVAL ENCOMPASSES WEIGHTS AND VERTICAL CG POSITIONS UP TO THE VALUES SHOWN.
- 3. THIS PREAPPROVAL WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION. COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR). USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS, AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 4. SEE GENERAL NOTES: SHEETS 1 AND 2



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER

DES. J. ROBERSON

JOB NO. 14-2307


DATE 10/4/23

SHEET

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS


CONCRETE SLAB ON METAL DECK

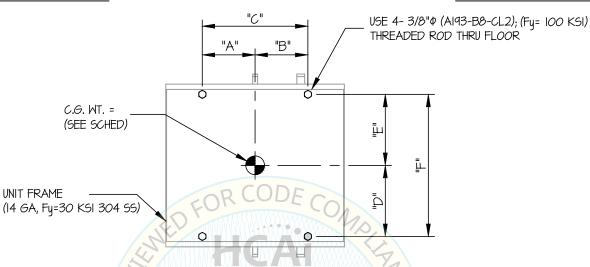

OPM-0082

UNIT	WEIGHT (Ib.)	"Y" (in.) [i "A" (in)t	"B" (in)	"C" (in.)	"D" (in.)	"E" (in.)	"F" (in.)	** Tu (lb.)	** Vu (lb.)
RELIANCE 300 XLS	646	35.88	8.88	8.12	17	11.25	12.25	23.5	2311	605
AMSCO 2532 SINGLE CHAMBER	1054 DA	31.3	18.5 2	4/8502	23 ₁₇	11.75/	11.75	23.5	3143	945

^{**} VALUES DO NOT INCLUDE $\Omega_{
m o}$

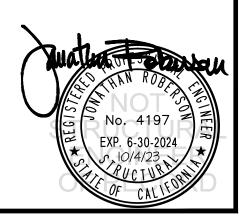

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING www.EquipmentAnchorage.com

STERIS CORPORATION

DES. J. ROBERSON

DATE


JOB NO. 14-2307

10/4/23

8 8

оғ 8 энеетз

AMSCO-2532 SINGLE CHAMBER WASHER/DISINFECTOR AND RELIANCE 300XLS GLASSWARE WASHER

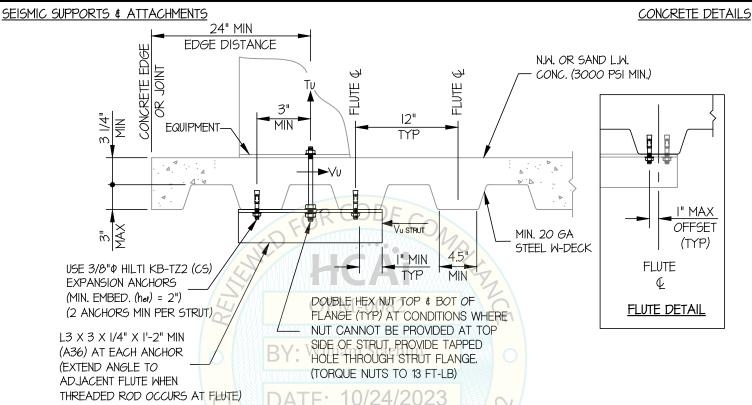

<u>MIN STEEL DECK REQUIREMENTS AND STR<mark>UT D</mark>ETAIL</u>

