

OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION

APPLICATION FOR OSHPD PREAPPROVAL	OFFICE USE ONLY									
OF MANUFACTURER'S CERTIFICATION (OPM)	APPLICATION #: OPM-0225-13									
OSHPD Preapproval of Manufacturer's Certification (OPM)										
Type: New Renewal Update to Pre-CBC 2013 OPA Number:										
Manufacturer Information										
Manufacturer: Sysmex America, Inc.										
Manufacturer's Technical Representative: Alan Burton										
Mailing Address: 577 Aptakisic Road, Lincolnshire, IL. 60069										
Telephone: (224) 543-9308 Email: Dburtor	na@sysmex.com									
Product Information	OMB									
Product Name: XN-1000 Analyzer System OSI / DO										
Product Type: Other electrical and mechanical components 13										
Product Model Number: N/A BY: William Stack	lin									
General Description: Automated hematology analyzer										
DATE: 01/26/2016	T C									
<u> </u>	N N N N N N N N N N N N N N N N N N N									
Applicant Information	\$\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\									
Applicant Company Name: EASE Co.	COX									
Contact Person: Jonathan Roberson, S.E.										
Mailing Address: 5877 Pine Ave. Suite 210, Chino Hills, CA. 91709										
Telephone: (909) 606-7622 Email: J.Rob I hereby agree to reimburse the Office of Statewide Health Paccordance with the California Administrative Code, 2013.	erson@EASECo.com Planning and Development review fees in									
Signature of Applicant:	Date: 4/16/15									
Title: Principal Engineer Company Name: EASE	Co.									

"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"

14/1/MM

os 🗆 pd

STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY OSH-FD-700 (REV 1/24/13)

Page 1 of 2



OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION

Registered Design Professional Preparing Engineering Recommendations										
Company Name: EASE Co.										
Name: Jonathan Roberson, S.E. California License Number: S4197										
Mailing Address: 5877 Pine Ave. Suite 210, Chino Hills, CA. 91709										
Telephone: 909-606-7667 Email: J.Roberson@EASECo.com										
OSHPD Special Seismic Certification Preapproval (OSP)										
 □ Special Seismic Certification is preapproved under OSP-(Separate application for OSP is required) □ Special Seismic Certification is not preapproved 										
Certification Method(s)										
☐ Testing in accordance with: ☐ ICC-ES AC156 ☐ FM 1950-10 ☐ Other* (Please Specify):										
*Use of test criteria other than those adopted by the California Building Standards Code, 2013 (CBSC 2013) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2013 may be used when approved by OSHPD prior to testing. Analysis										
Signature: Date: 01/26/2016										
Print Name: William Staehlin										
Title: SSE										
Condition of Approval (if applicable):										

"Access to Safe, Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"

Page 2 of 2



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

Office of Statewide Health Planning and Development PREAPPROVAL OF MANUFACTURER'S CERTIFICATION OPM-0225-13

THIS PREAPPROVAL CONFORMS TO THE 2013 CALIFORNIA BUILDING CODE

MANUFACTURER: SYSMEX

Sheet: 1 of 8

EQUIPMENT NAME: XN-1000 SERIES

Date: 12/21/15

GENERAL NOTES

- 1. THIS OSHPD PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2013 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2013 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2013 CALIFORNIA BUILDING CODE WHERE Sps IS NOT GREATER THAN 2.20.
- 4. FORCES PER ASCE 7-10 SECTION 13.3.1, EQUATIONS 13.3-1, 13.3-2 & 13.3-3, WHERE SDS 2.20, ap = 1.0, lp = 1.5, Rp = 1.5, z/h = 0 AT CONCRETE SLAB & z/h < 1 AT CONCRETE SLAB ON METAL DECK. SEE FOLLOWING SHEETS FOR Ω₀
- 5. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- 6. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. CONCRETE SLAB ON METAL DECK DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING. (i.e. z/h < 1)
- 8. CONCRETE SLAB ON GRADE DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION BELOW GRADE. (i.e. z/h = 0)

9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE TO SUPPORT WEIGHTS AND FORCES SHOWN IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2013 CBC AND WITH THE DETAILS,
 MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE
 PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT PROJECT SPECIFIC VALUES OF SDS & z/h RESULT IN SEISMIC FORCES (Eh, Ev) THAT DO NOT EXCEED THE VALUES ON THE DETAILS.
- D. VERIFY THAT THE CONCRETE SLAB TO WHICH THE EQUIPMENT IS ANCHORED MEETS THE REQUIREMENTS OF THE APPLICABLE ICC ESR.
- E. VERIFY THAT THE ANCHORS ARE AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPENINGS (SEE TYPICAL DETAIL ON SHEET 2).
- F. VERIFY THAT ALL NEW OR EXISTING ANCHORS ARE AN ADEQUATE DISTANCE FROM THE UNIT ATTACHMENTS AND CHECK FOR INTERACTION WHERE OTHER ANCHORS ARE WITHIN 18" OR 6hef FROM THIS UNIT'S ANCHORS.



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

12/21/15

www.EquipmentAnchorage.com

SYSMEX DES. J. ROBERSON

ЈОВ NO. 11-1443

DATE

2

of 8 sheets


XN-1000 SERIES

10. EXPANSION ANCHORS:


A. ATTACHMENT IS TO BE MADE WITH THE ANCHORS LISTED BELOW AND INSTALLED AS DESCRIBED IN THE CORRESPONDING ICC REPORT.

Anchor Diameter	Concrete Type	Min. f'c (psi)	Anchor Type	ICC Report No.	Min. Embed.	Min. Spacing	Min. Edge Dist.	Min. Conc. Thickness	Torque Test	Direct Tension
3/8"	Sand Light Weight	3000	Hilti Kwik Bolt TZ	ESR-1917	2"	N/A	N/A	See Sheet 7 of 8	25 FT-LB	1186 lb
1/2"	Normal Weight	3000	Hilti Kwik Bolt TZ	ESR-1917	2"	3"	9"	4"	40 FT-LB	1204 lb

- B. THIS PREAPPROVAL ALLOWS FOR UP TO A MAXIMUM OF 2 ADJACENT CONCRETE SLAB EDGES, 9" AWAY MINIMUM (i.e. CORNER). SEE ADJACENT DETAIL FOR ADDITIONAL MINIMUM ALLOWABLE CONCRETE EDGE DISTANCES.
- C. TESTING OF EXPANSION ANCHORS PER 2013 CBC, 1913A.7:
 TESTING SHALL BE DONE IN THE PRESENCE OF THE SPECIAL
 INSPECTOR AND A REPORT OF THE TEST RESULTS SHALL BE
 SUBMITTED TO OSHPD
 - (i) AFTER AT LEAST 24 HOURS HAVE ELAPSED SINCE INSTALLATION, DIRECT PULL TENSION TEST OR TORQUE TEST AT LEAST 50% OF THE ANCHORS.
 - (ii) ACCEPTANCE CRITERIA:
 - DIRECT TENSION TEST: THE ANCHOR SHOULD HAVE NO OBSERVABLE MOVEMENT AT THE TEST LOAD. A PRACTICAL WAY TO DETERMINE OBSERVABLE MOVEMENT IS THAT THE WASHER BECOMES LOOSE.
 - TORQUE TEST: THE APPLICABLE TORQUE MUST BE ACHIEVED
 WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: 1/2 TURN OF THE
 NUT
 - (iii) IF ANY ANCHOR FAILS, TEST ALL ANCHORS.
- 11. BOLTS THROUGH CONCRETE ON METAL DECK
 - A. BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUTS AFTER THE SNUG TIGHT (THE SNUG-TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQUIRED TO BRING THE CONNECTED PLIES INTO FIRM CONTACT) CONDITION IS ACHIEVED, UNLESS OTHERWISE NOTED.
 - B. THROUGH BOLT HOLES SHALL BE 1/16" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE + 1/16) FOR CONCRETE.
 - C. THROUGH-BOLTS IN CONCRETE SHALL RECEIVE SPECIAL INSPECTION AND TESTING (THROUGH BOLTS WITH STEEL TO STEEL CONNECTION IN TENSION DO NOT REQUIRE TENSION TESTING) IN ACCORDANCE WITH REQUIREMENTS FOR POST-INSTALLED ANCHORS.

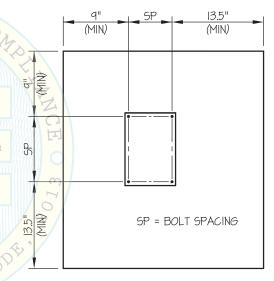

TYPICAL CONCRETE EDGE DETAIL

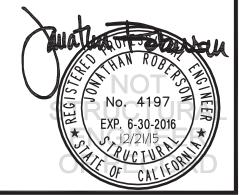

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING www.EquipmentAnchorage.com

DES. J. ROBERSON


SYSMEX

11-1443 JOB NO.

XN-1000 SERIES

12/21/15 DATE

SHEETS

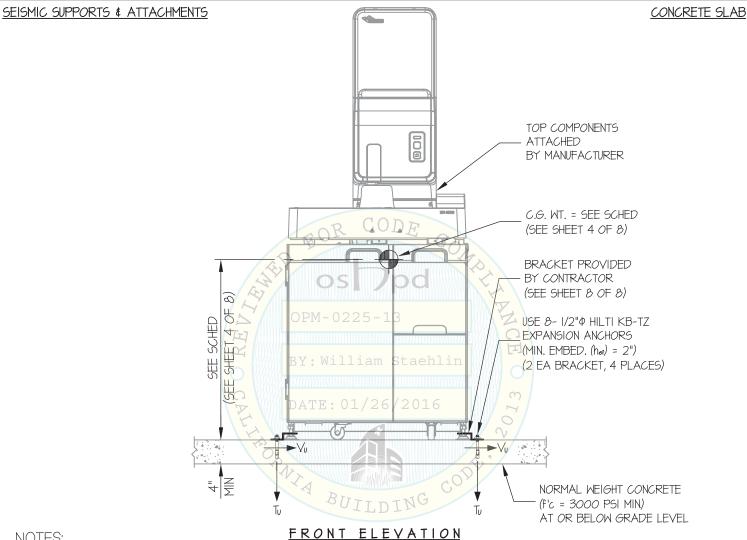

1. FORCES ARE DETERMINED PER 2013 CALIFORNIA BUILDING CODE AND ASCE 7-10 STRENGTH DESIGN IS USED. (SDS = 2.20, Δp = 1.0, |p| = 1.5, R_p = 1.5, Ω_0 = 1.5, z/h = 0)

> HORIZONTAL FORCE (En) = 0.99 Wp HORIZONTAL FORCE (Emh) = 1.49 Wp (FOR CONCRETE ANCHORAGE) VERTICAL FORCE (Ev) = 0.44 Wp

- 2. CENTER OF GRAVITY (C.G.) AND WEIGHT ARE THE GOVERNING PARAMETERS FOR DESIGN, THIS PREAPPROVAL ENCOMPASSES ALL WEIGHTS UP TO THE MAXIMUM WEIGHT SHOWN.
- 3. STRUCTURAL ENGINEER OF RECORD FOR THE BUILDING SHALL PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 4. SFF GENERAL NOTES: SHEET 1 AND 2



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

SYSMEX

DES. J. ROBERSON 11-1443 JOB NO.


XN-1000 SERIES

12/21/15 DATE

OF SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB

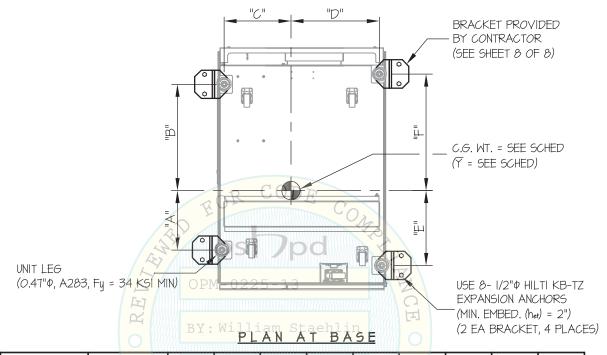

	MODEL	WEIGHT (lb.)	CG (in.)	<u>"</u> "A" (in.)	/"B"(in.)2	"C" (in.)	"D" (in.)	,"E" (in.)	"F" (in.)	* Tu (lb.)	* Vu (lb.)
	XN-1000+WG-10	359	25.67	9.1	16.3	10.2	13.5	11.4	17.9	603	171
	XN-1000+WG-11	350	27.47	14.2	15.7	11.7	10.4	14.2	15.7	555	138
* VALUES INCLUDE Ω_0											



SYSMEX

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

DES. J. ROBERSON

11-1443 JOB NO.


12/21/15

DATE

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

XN-1000 SERIES

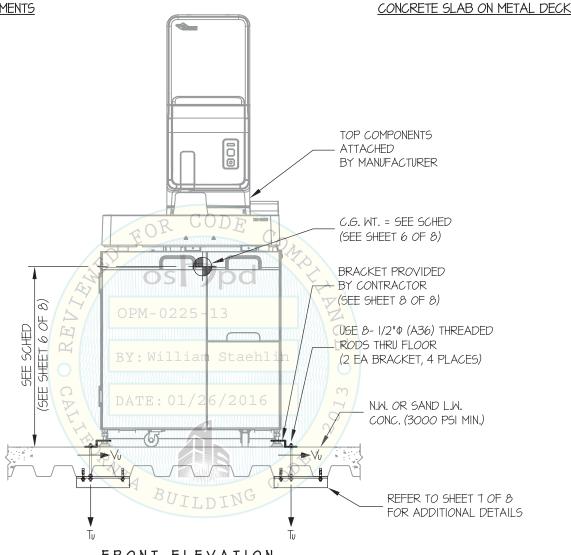

NOTES:

FRONT ELEVATION

 FORCES ARE DETERMINED PER 2013 CALIFORNIA BUILDING CODE AND ASCE 7-10. STRENGTH DESIGN IS USED. (SDS = 2.20, Δp = 1.0, |p| = 1.5, Rp = 1.5, Ω_0 = 1.5, z/h < 1)

> HORIZONTAL FORCE (En) = 2.64 Wp HORIZONTAL FORCE (Emh) = 3.96 Wp (FOR CONCRETE ANCHORAGE VERTICAL FORCE (Ev) = 0.44 Wp

- 2. CENTER OF GRAVITY (C.G.) AND WEIGHT ARE THE GOVERNING PARAMETERS FOR DESIGN, THIS PREAPPROVAL ENCOMPASSES ALL WEIGHTS UP TO THE MAXIMUM WEIGHT SHOWN.
- 3. STRUCTURAL ENGINEER OF RECORD FOR THE BUILDING SHALL PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

SYSMEX

DES. J. ROBERSON 11-1443

XN-1000 SERIES


12/21/15 DATE

JOB NO.

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

OF CONCRETE SLAB ON METAL DECK

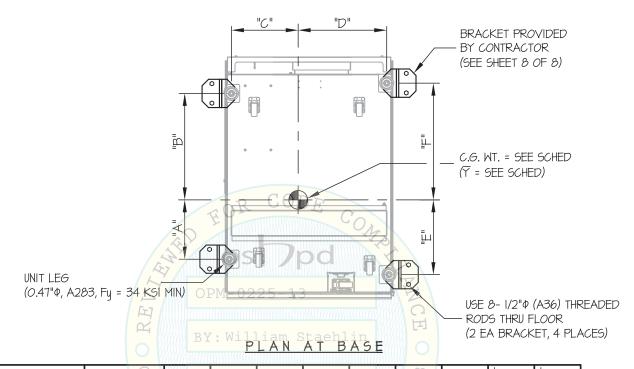

	MODEL	WEIGHT (lb.)	CG (in.)	747 (in.)	1"B"2(n.)/	20" (in)	"D" (in.)	"E" (in.)	"F" (in.)	* Tu (lb.)	* Vu (lb.)
	XN-1000+WG-10	359	25.67	9.1	16,3	10.2	13.5	<u>11.4</u>	17.9	1143	304
	XN-1000+WG-11	350	27.47	14.2	15.7	11.7	10,4	14.2	15.7	1040	245
* VALUES DO NOT INCLUDE Ω_0 $\begin{array}{c} & & & & & & & & & & & & \\ & & & & & & $											



EASE

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

OF

SYSMEX

DES. J. ROBERSON

11-1443


_

XN-1000 SERIES

DATE 12/21/15

JOB NO.

8 SHEETS

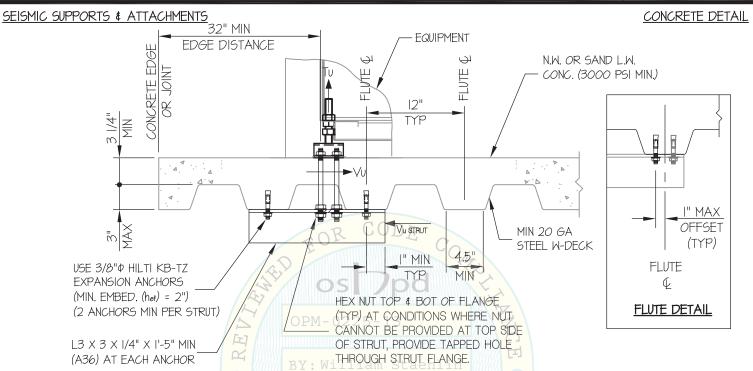

MIN STEEL DECK REQUIREMENTS AND STRUT DETAIL

DATE: 01/26/2016

BUILDING



