

DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION OFFICE OF STATEWIDE HOSPITAL PLANNING AND DEVELOPMENT

APPLICATION FOR HCAI PREAPPROVAL OF	OFFICE USE ONLY					
MANUFACTURER'S CERTIFICATION (OPM)	APPLICATION #: OPM-0445					
HCAI Preapproval of Manufacturer's Certification (OPM)						
Type: New X Renewal/Update						
Manufacturer Information						
Manufacturer: Steris Canada ULC						
Manufacturer's Technical Representative: Wael Abdul-Hadi						
Mailing Address: 490 Boul, Armand-Paris, Quebec, QC G1C8A3						
Telephone: (440) 392-7688 Email: Wael_Abdul-Hadi@s	steris.com					
Product Information						
Product Name: AMSCO 7052 HP AND 7053HP WASHERS / DISINFECTORS	G					
Product Type: other mechanical or electrical components						
Product Model Number: 7052HP, 7053HP BY: William Staehlin						
General Description: washing/disinfecting of medical instruments and equipme	nt					
PATE: 04/19/2024	202					
Applicant Information						
Applicant Company Name: EASE LLC.						

Contact Person: Tiffany Tonn

Mailing Address: 1515 FAIRVIEW AVE, STE 205, MISSOULA, MT 59801

Telephone: (406) 541-3273 Email: tiffany@easeco.com

Title: Office Assistant

"A healthier California where all receive equitable, affordable, and quality health care"





DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION OFFICE OF STATEWIDE HOSPITAL PLANNING AND DEVELOPMENT

Registered Design Professonal Preparing Engineering Recommendations
Company Name: EASE LLC
Name: Jonathan Roberson California License Number: S4197
Mailing Address: 5877 Pine Ave., Suite 210, Chino Hills, CA 91709
Telephone: (951) 295-1892
HCAI Special Seismic Certification Preapproval (OSP)
_
Special Seismic Certification is preapproved under OSP OSP Number:
FOR CODE CO
Certification Method
Testing in accordance with:
Other(s) (Please Specify):
*Use of criteria other than those adopted by the California Building Standards Code, 2022 (CBSC 2022) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2022 may be used when approved by HCAI prior to testing.
X Analysis
Experience Data DATE: 04/19/2024
Combination of Testing, Analysis, and/or Experience Data (Please Specify):
OPVIA CODE
HCAI Approval
Date: 4/19/2024
Name: William Staehlin Title: Senior Structural Engineer
Condition of Approval (if applicable):

"A healthier California where all receive equitable, affordable, and quality health care"

HCAi

STATE OF CALIFORNIA - HEALTH AND HUMAN SERVICES AGENCY



5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

The Department of Health Care Access and Information
PREAPPROVAL OF MANUFACTURER'S CERTIFICATION
OPM-0445

THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE

MANUFACTURER:

STERIS CORPORATION

Sheet: 1 of 8

EQUIPMENT NAME:

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS

Date: 4/15/24

GENERAL NOTES

- 1. THIS HCAI PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2022 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2022 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE WHERE SDS IS NOT GREATER THAN 2.30.
- 4. FORCES PER ASCE 7-16 SECTION 13.3.1, EQUATIONS 13.3-1, 13.3-2 & 13.3-3, WHERE SDS = 2.30, \mathbf{a}_p = 1.0, \mathbf{I}_p = 1.5, \mathbf{r}_p = 1.5,
- 5. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- 6. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. CONCRETE SLAB ON METAL DECK DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING. (i.e. z/h < 1)
- 8. CONCRETE SLAB DETAIL VALID FOR DEMANDS SHOWN AT OR BELOW GRADE. (i.e. z/h = 0)

9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE TO SUPPORT WEIGHTS AND FORCES SHOWN IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2022 CBC AND WITH THE DETAILS, MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT PROJECT SPECIFIC VALUES OF SDS & z/h RESULT IN SEISMIC FORCES (Eh, Ev) THAT DO NOT EXCEED THE VALUES ON THE DETAILS.
- D. VERIFY THAT THE CONCRETE SLAB TO WHICH THE EQUIPMENT IS ANCHORED MEETS THE REQUIREMENTS OF THE APPLICABLE ICC ESR REPORT. AND THIS OPM.
- E. VERIFY THAT THE ANCHORS ARE AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPENINGS (SEE TYPICAL DETAIL ON SHEET 2).
- F. VERIFY THAT ALL NEW OR EXISTING ANCHORS ARE AN ADEQUATE DISTANCE FROM THE UNIT ATTACHMENTS AND CHECK FOR INTERACTION WHERE OTHER ANCHORS ARE WITHIN 18" OR 6hef FROM THIS UNIT'S ANCHORS.



4/15/24

DATE

www.EquipmentAnchorage.com

SHEETS

STERIS CORPORATION

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS


JOB NO. 14-2402

10. EXPANSION ANCHORS:

A. ATTACHMENT IS TO BE MADE WITH THE ANCHORS LISTED BELOW AND INSTALLED AS DESCRIBED IN THE CORRESPONDING ICC REPORT.

Anchor Diameter	Concrete Type	Min. f'c (psi)	Anchor Type	nchor Type ICC Min. Report No. Embed.				Min. Conc. Thickness	Torque Test	Direct Tension Test	
1/2"	Sand Light Weight	3000 Hilti Kwik Bolt TZ (CARBON STEEL)		ESR-4266	2.5"	7.50"	12"	See Detail "A"	50 FT-LB	N/A	
3/4"	Normal Weight	3000	Hilti Kwik Bolt TZ2 (STAINLESS STEEL)	ESR-4266	3.25"	15"	34"	6"	125 FT-LB	5090 lb	

- B. THIS PREAPPROVAL ALLOWS FOR UP TO A MAXIMUM OF 2 ADJACENT CONCRETE SLAB EDGES, 34" AWAY MINIMUM (i.e. - CORNER).
 SEE ADJACENT DETAIL FOR ADDITIONAL MINIMUM ALLOWABLE CONCRETE EDGE DISTANCES.
- C. TESTING AND SPECIAL INSPECTION OF EXPANSION ANCHORS SHALL
 BE PERFORMED BY AN APPROVED INDEPENDENT AGENCY
 EMPLOYED BY THE FACILITY OWNER PER CBC 1704A & 1910A.5
 AND CAC 7-149. ALL REPORTS SHALL BE SENT TO THE INSPECTOR
 OF RECORD, OWNER AND THE ARCHITECT OR ENGINEER IN
 RESPONSIBLE CHARGE.
 - (i) AFTER AT LEAST 24 HOURS HAVE ELAPSED SINCE INSTALLATION, DIRECT PULL TENSION TEST OR TORQUE TEST AT LEAST 50% OF THE ANCHORS.
 - (ii) ACCEPTANCE CRITERIA:
 - DIRECT TENSION TEST: THE ANCHOR SHOULD HAVE NO OBSERVABLE MOVEMENT AT THE TEST LOAD. A PRACTICAL WAY TO DETERMINE OBSERVABLE MOVEMENT IS THAT THE WASHER BECOMES LOOSE.
 - TORQUE TEST: THE APPLICABLE TORQUE MUST BE ACHIEVED WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: 1/2 TURN OF THE NUT
 - (iii) IF ANY ANCHOR FAILS, TEST ALL ANCHORS.
- D. AVOID DAMAGING EXISTING STEEL REINFORCING IN CONCRETE SLAB WHEN INSTALLING CONCRETE EXPANSION ANCHORS.
- E. PROVIDE FOR FULL THREAD ENGAGEMENT OF NUT & WASHER.
- 11. BOLTS THROUGH CONCRETE ON METAL DECK
 - A. BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUTS AFTER THE SNUG TIGHT (THE SNUG-TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQUIRED TO BRING THE CONNECTED PLIES INTO FIRM CONTACT) CONDITION IS ACHIEVED, UNLESS OTHERWISE NOTED.
 - B. THROUGH BOLT HOLES SHALL BE 1/16" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE + 1/16) FOR CONCRETE.
 - C. THROUGH-BOLTS IN CONCRETE SHALL RECEIVE SPECIAL INSPECTION AND TESTING (THROUGH BOLTS WITH STEEL TO STEEL CONNECTION IN TENSION DO NOT REQUIRE TENSION TESTING) IN ACCORDANCE WITH REQUIREMENTS FOR POST-INSTALLED ANCHORS.

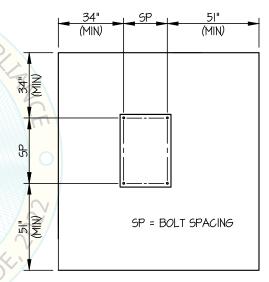

TYPICAL CONCRETE EDGE DETAIL

(SLAB ON GRADE ONLY)



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS DES. J. ROBERSON


JOB NO. 14-2402

DATE 4/15/24

3

- 8 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

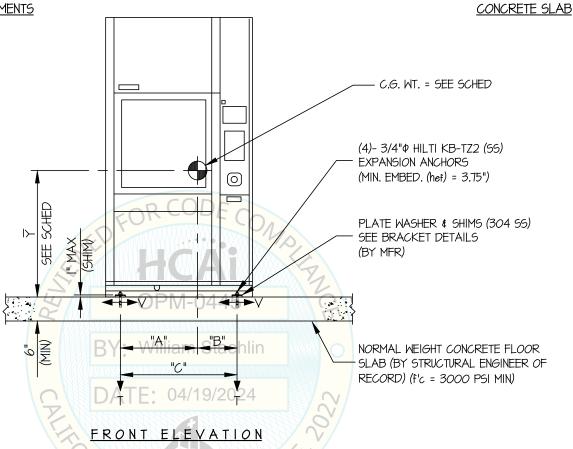

NOTES:

1. FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: SDS = 2.30, 20 = 1.0, 10 = 1.5, 10 = 1.5, 10 = 2.0, 10 = 2.0, 10 = 2.0, 10 = 1.5, 10 = 2.0, 10

HORIZONTAL FORCE (Eh) = 1.035Wp

HORIZONTAL FORCE (Emh) = 2.07 Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (Ev) = 0.46 Wp

- 2. THIS PREAPPROVAL ENCOMPASSES WEIGHTS AND VERTICAL CG POSITIONS UP TO THE VALUES SHOWN.
- 3. THIS PREAPPROVAL WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION. COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR). USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS, AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 5. SEE GENERAL NOTES; SHEETS 1 AND 2



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS DES. J. ROBERSON

JOB NO. 14-2402

DATE


4/15/24

of 8 SHEETS

SHEET


SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB

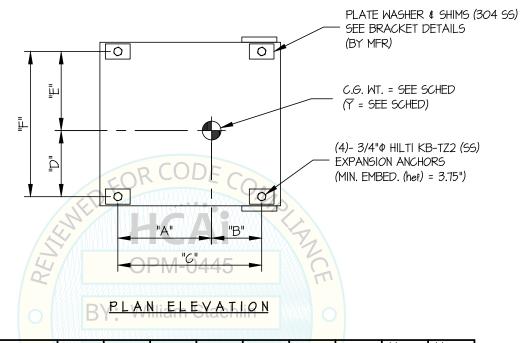

MODELS	WEIGHT (lb.)	₹ (in.)	"A" (in.)	"B"4(in.)	9"C" (in.)4	"D" (in.)	"E" (in.)	"F" (in.)	Tu (lb.)	++ Vu (lb.)
AMSCO 7052 HP	1579	36.0	21.74	11.69	33.43	12.91	14.81	27.72	3083	1325
AMSCO 7053 HP	1661	37.0	21.77	11.66	33,43	15.33	18,39	33.72	2820	1401

⁺⁺ VALUES INCLUDE $\Omega_{
m o}$

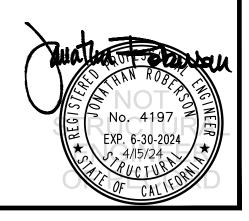

www.EquipmentAnchorage.com

STERIS CORPORATION


AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS DES. J. ROBERSON


JOB NO. 14-2402

DATE 4/15/24

5

8 SHEETS

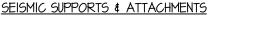

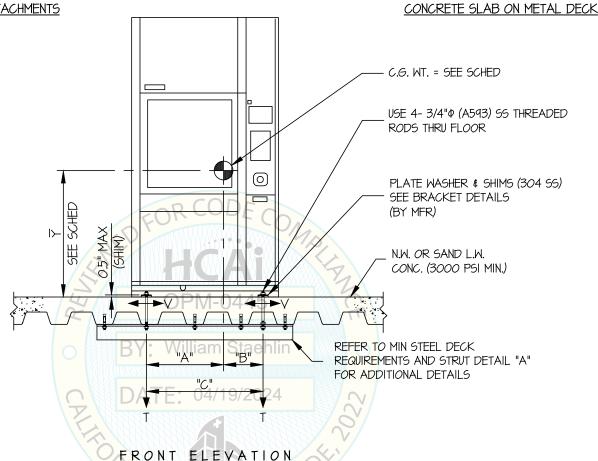

NOTES:

1. FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: SDS = 2.30, 2p = 1.0, 1p = 1.5, 1p

HORIZONTAL FORCE (Eh) = 2.76 Wp

HORIZONTAL FORCE (Emh) = 5.52 Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (Ev) = 0.46 Wp

- 2. THIS CALCULATION ENCOMPASSES WEIGHTS AND VERTICAL C.G. POSITIONS NOT EXCEEDING VALUES SHOWN.
- 3. THIS CALCULATION WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION, COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR), USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS, AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 5. SEE GENERAL NOTES: SHEETS 1 AND 2



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS DES. J. ROBERSON

JOB NO. 14-2402

DATE

4/15/24


O

SHEET

ог 8 знеетз

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

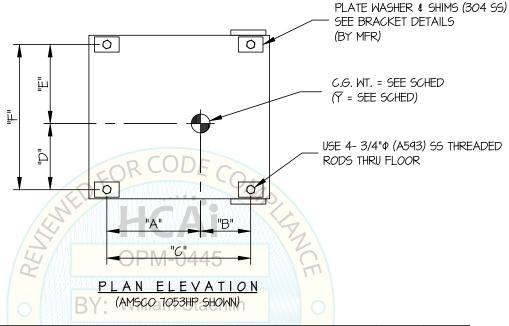

MODELS	WEIGHT (lb.)	√ (in.)	"A" (in.)	"B" (in.)	"C" (in.)4	"D" (in.)	"E" (in.)	"F" (in.)	++ Tu (lb.)	++ Vu (lb.)
AMSCO 7052 HP	1579	35.5	21.74	11.69	33.43	12.91	14.81	27.72	4130	1766
AMSCO 7053 HP	1661	36.5	21.77	11.66	33.43	15.33	18.39	33.72	3791	1868

++ VALUES INCLUDE $\Omega_{\rm o}$



www.EquipmentAnchorage.com

STERIS CORPORATION

AMSCO 7052 HP AND 7053 HP WASHERS / DISINFECTORS

DES. J. ROBERSON

JOB NO. 14-2402

DATE 4/15/24


7

8 SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

THREADED ROD OCCURS AT FLUTE)

CONCRETE SLAB ON METAL DECK

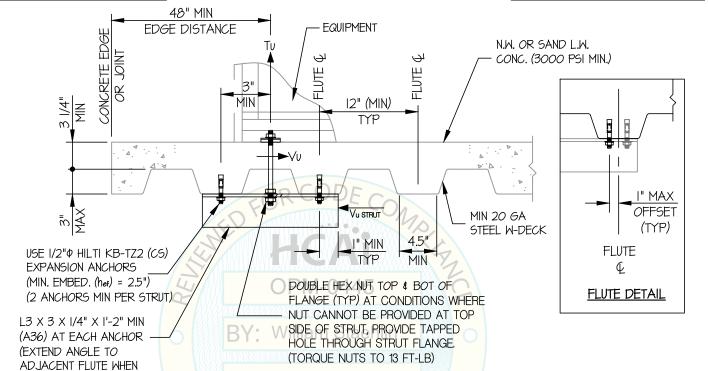

MIN STEEL DECK REQUIREMENTS AND STRUT DETAIL

DATE: 04/19/2024

PANIA BUIL





EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING www.EquipmentAnchorage.com SHEET DE8. J. ROBERSON STERIS CORPORATION 14-2402 JOB NO. AMSCO 7052 HP AND 7053 HP **WASHERS / DISINFECTORS** 4/15/24 DATE SHEETS SEISMIC SUPPORTS & ATTACHMENTS **BRACKET DETAILS** (1) 0.813¢ HOLE SIDE WASHER William Staehl 04/19/202 0.375"



SHIMS

13/16" PHOLES

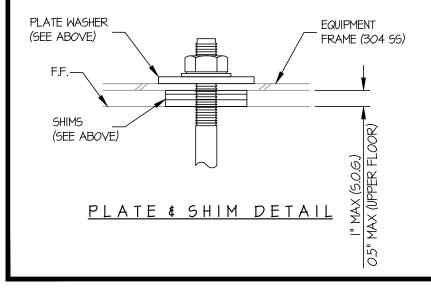

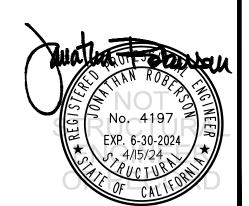

VARIES THICKNESS

2.50 SHIMS

(304 SS TYP)