

# DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION OFFICE OF STATEWIDE HOSPITAL PLANNING AND DEVELOPMENT

| APPLICATION FOR HCAI PREAPPROVAL OF  | OFFICE USE ONLY  |  |  |  |  |
|--|--|--|--|--|--|
| MANUFACTURER'S CERTIFICATION (OPM)   | APPLICATION #: OPM-0537  |  |  |  |  |
| HCAI Preapproval of Manufacturer's Certification (OPM)                               |  |  |  |  |  |
| Type: New X Renewal/Update   |  |  |  |  |  |
| Manufacturer Information   |  |  |  |  |  |
| Manufacturer: Sysmex America, Inc.   |  |  |  |  |  |
| Manufacturer's Technical Representative: Jason Anderson                              |  |  |  |  |  |
| Mailing Address: 577 Aptakisic Road, Lincolnshire, IL 60669                          |  |  |  |  |  |
| Telephone: (801) 703-1487 Email: andersonja@sysme                                    | ex.com   |  |  |  |  |
| ED FOR CODE CON  |  |  |  |  |  |
| Product Information  |  |  |  |  |  |
| Product Name: UN Series System   | 12   |  |  |  |  |
| Product Type: Automated Urinalysis System  | THE STATE OF THE S |  |  |  |  |
| Product Model Number: Components: Clinitek Novus, UF-5000, UD-10, ST-10 WG-45, WG-46 | , ST-1 <mark>1, ST</mark> -12, TH-11; Supports: WG-13, WG-44,  |  |  |  |  |
| General Description: Fully Automated Urine Analyzers                                 |  |  |  |  |  |
| DATE: UTITZIZUZ4   | 200  |  |  |  |  |
| Applicant Information  |  |  |  |  |  |
| Applicant Company Name: Sysmex America, Inc.   |  |  |  |  |  |
| Contact Person: Jason Anderson   |  |  |  |  |  |

Title: Sr. Product Manager

Telephone: (801) 703-1487

Mailing Address: 577 Aptakisic Road, Lincolnshire, IL 60669

"A healthier California where all receive equitable, affordable, and quality health care"

STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY



Email: andersonja@sysmex.com



# DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION OFFICE OF STATEWIDE HOSPITAL PLANNING AND DEVELOPMENT

| Registered Design Professonal Preparing Engineering Recommendations   |
|---|
| Company Name: CYS STRUCTURAL ENGINEERS, INC.  |
| Name: Dieter Siebald California License Number: S4346   |
| Mailing Address: 2495 Natomas Park Drive, Suite 650, Sacramento, CA 95833   |
| Telephone: (916) 920-2020 Email: dieters@cyseng.com   |
|   |
|   |
| HCAI Special Seismic Certification Preapproval (OSP)  |
| Special Seismic Certification is preapproved under OSP OSP Number:  |
| OR CODE O   |
| Certification Method  |
| Certification Method  |
| Testing in accordance with: ICC-ES AC156 FM 1950-16   |
| Other(s) (Please Specify):  |
| *Use of criteria other than those adopted by the California Building Standards Code, 2022 (CBSC 2022) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2022 may be used when approved by HCAI prior to testing. |
| X Analysis  |
| Experience Data  DATE: 07/12/2024   |
| Combination of Testing, Analysis, and/or Experience Data (Please Specify):  |
|   |
| HCAI Approval   |
| Date: 7/12/2024   |
| Name: Timothy Piland Title: Senior Structural Engineer  |
| Condition of Approval (if applicable):  |

"A healthier California where all receive equitable, affordable, and quality health care"

**HCA**i

STATE OF CALIFORNIA - HEALTH AND HUMAN SERVICES AGENCY

### TABLE OF CONTENTS OPM-0537

|   | PAGE                         |
|---|------------------------------|
| GENERAL NOTES   | . 2                          |
| ABBREVIATIONS   | 4                            |
| DESIGN CRITERIA   | 5                            |
| CONFIGURATION EXAMPLES CONFIGURATION TABLE  | 6<br>8                       |
| MODULE #1 - PLAN VIEW & ELEVATION MODULE #2 - PLAN VIEW & ELEVATION MODULE #3 - PLAN VIEW & ELEVATION MODULE #4 - PLAN VIEW & ELEVATION MODULE #5 - PLAN VIEW & ELEVATION MODULE #6 - PLAN VIEW & ELEVATION | 11<br>12<br>13               |
| COMPONENT CONNECTIONS TO SUPPORT WAGON FRAMES LEVELING LEG DETAILS BRACKET LOCATIONS BRACKET DETAILS BRACKET DETAILS  | . 21<br>. 24<br>. 25<br>. 26 |
| ATTACHMENT DETAILS THRU CONC FILL OVER MTL DECK (CASE 1) ATTACHMENT DETAILS INTO CONC FILL OVER MTL DECK (CASE 2) ATTACHMENT DETAILS TO 4" CONC SLAB ON GRADE (CASE 3) 4                                    | . 30                         |
| MODULE #5 ALTERNATE OPTION - PLAN VIEW & ELEVATION  | 32                           |

#### NOTES:

- THESE DRAWINGS ARE PREPARED FOR SYSMEX AMERICA, INC., LINCOLNSHIRE, ILLINOIS.
- 2. THE CONTRACTOR & INSPECTOR OF RECORD SHALL OBTAIN A COPY OF THIS PRE-APPROVAL FROM THE STATE OF CALIFORNIA HEALTH CARE ACCESS AND INFORMATION (HCAI/OSHPD) PRE-APPROVAL PROGRAMS WEBSITE.
- 3. THIS PRE-APPROVAL COVERS THE SUPPORTS & ATTACHMENTS OF THE EQUIPMENT TO THE SUPPORTING STRUCTURE. THE EQUIPMENT, SUPPORT & ATTACHMENT HARDWARE ARE SUPPLIED BY THE MANUFACTURER. THE EXPANSION ANCHORS, THRU-BOLTS & STRUT PLATES SHOWN IN THIS OPP SHALL BE SUPPLIED & INSTALLED BY THE CONTRACTOR.

SHEET TITLE: TABLE OF CONTENTS



CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 www.cyseng.com Page: 1 of 33

### **GENERAL NOTES:**



1. THIS HCAI PRE-APPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE CBC 2022. THE DEMAND (DESIGN FORCES) FOR USE W/ THIS OPM SHALL BE BASED ON THE CBC 2022.

2. IT IS THE RESPONSIBILITY OF THE SEOR FOR A SITE SPECIFIC PROJECT TO VERIFY:

**UN SERIES SYSTEM** 

- THE ADEQUACY OF THE NEW OR EXISTING STRUCTURE TO RESIST THE FORCES & WT SPECIFIED FOR EA EQUIP IN ADDITION TO ALL OTHER LOADS. PROVIDE & DESIGN SUPPLEMENTARY MEMBERS AS REQ.
- THAT THE FLR ANCHORS ARE LOCATED AT AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPGS.
- THAT THE FLR ANCHORS ARE LOCATED AT AN ADEQUATE DISTANCE FROM ANY NEW OR EXISTING ANCHORS. THE SPCG SHOWN IN THE EXPANSION ANCHOR TABLE ON PG 3 IS THE REQ MIN SPCG OF THE GIVEN DIA ANCHORS. THE REQ SPCG FROM ANCHORS OF OTHER DIAMETERS & EMBEDMENTS MAY VARY & SHALL BE EVALUATED BY THE SEOR.
- THAT THE INSTALLATION IS IN CONFORMANCE W/ THE CBC 2022 & W/ THE DETAILS SHOWN IN THIS PRE-APPROVAL.
- THAT THE ACTUAL EQUIP'S WT, CENTER OF GRAVITY (CG) LOCATION, ANCHOR LOCATIONS, ANCHOR DETAILS, & THE MATERIAL & GAGE OF THE EQUIP WHERE ATTACHMENTS ARE MADE, AGREE W/ THE INFO SHOWN ON THE PRE-APPROVAL DOCUMENTS.
- THAT THE CONC SLAB TO WHICH THE EQUIP IS ANCHORED SHALL MEET THE REQUIREMENTS OF THE APPLICABLE ICC REPORT & THIS OPM.
- 3. EXPANSION ANCHORS INSTALLED IN NWC OR SLWC SHALL BE STAINLESS STEEL HILTI KB-TZ2 EXPANSION ANCHORS AS NOTED COMPLYING W/ ESR-4266 REISSUED DECEMBER 2023.
  - A. INSTALLATION: INSTALL THE EXPANSION ANCHORS IN ACCORDANCE W/ THE REQUIREMENTS GIVEN IN THE ICC EVALUATION REPORT FOR THE SPECIFIC ANCHOR & THE PARAMETERS GIVEN IN THE EXPANSION ANCHOR TABLE ON PG 3.
  - JOB TESTING: FOR VERIFYING SATISFACTORY INSTALLATION WORKMANSHIP, PERFORM JOBSITE TESTING IN ACCORDANCE W/ THE EXPANSION ANCHOR TABLE PROVIDED IN THIS DOCUMENT. TORQUE TEST 50% OF THE INSTALLED ANCHORS. ALL TESTS SHALL BE CONDUCTED IN THE PRESENCE OF THE SPECIAL INSPECTOR & REPORT OF TEST RESULTS SHALL BE SUBMITTED TO HCAL. IF ANY ANCHOR FAILS THE TEST, TEST ALL ANCHORS. THE TEST SHALL BE PERFORMED 24 HOURS OR MORE AFTER INSTALLATION. TESTING MAY BE DONE PRIOR TO EQUIP INSTALLATION, HOWEVER NUT SHALL BE RETORQUED TO INSTALLATION TORQUE AFTER EQUIPMENT INSTALL. ALSO REFER TO 2022 CBC 1910A.5 "TESTS FOR POST-INSTALLED ANCHORS IN CONCRETE". REPORT OF TEST RESULTS SHALL BE SUBMITTED TO HCAI.
  - C. FAILURE/ACCEPTANCE CRITERIA: THE FOLLOWING CRITERIA APPLY FOR THE ACCEPTANCE OF INSTALLED **ANCHORS:**
  - TORQUE WRENCH METHOD: THE APPLICABLE TEST TORQUE MUST BE REACHED WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: ONE-HALF (½) TURN OF THE NUT.

    D. AVOID DAMAGING (E) STL REINF IN CONC SLAB WHEN INSTALLING CONC EXPANSION ANCHORS.

  - PROVIDE FOR FULL THRD ENGAGEMENT OF NUT & WASHER.
- 4. BOLTS THRU CONC ON MTL DECK:
  - BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUT AFTER SNUG TIGHT CONDITION IS ACHIEVED, UNO. THE SNUG TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQ TO BRING THE CONNECTED PLIES INTO FIRM CONTACT.
  - THRU-BOLT HOLES SHALL BE 1/6" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE +  $\frac{1}{16}$ ").
  - THRU-BOLTS IN CONC SHALL RÉCEIVE SPECIAL INSPECTION & TESTING IN ACCORDANCE W/ REQUIREMENTS FOR POST-INSTALLED ANCHORS. THRU-BOLTS W/ STL TO STL CONN IN TENSION DO NOT REQUIRE TESTING.

SHEET TITLE: GENERAL NOTES



CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

TEL (916) 920-2020 Date:

Job No: 06-10-2024

4 of 35

23050

Dimscale: 12

Time:Jul11,2024-03:40pm Login:camachom

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg


www.cyseng.com Page:

2 of 33

OF CALIFOR



#### **GENERAL NOTES CONTINUED:**

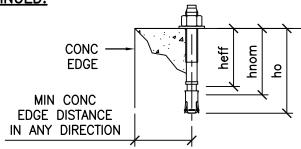

#### **EXPANSION ANCHOR TABLE**

| CONDITION<br>OF<br>ANCHORAGE | ANCHOR<br>DIA &<br>TYPE<br>(INCH) | INSTALLATION<br>EMBED<br>(INCH)<br>hnom | EFFECTIVE<br>EMBED<br>(INCH)<br>hef | HOLE<br>DEPTH<br>(INCH)<br>ho | MIN CONC<br>THK<br>(INCH)<br>h | MIN CONC<br>EDGE<br>DISTANCE<br>(INCH) | MIN<br>ANCHOR<br>SPCG<br>(INCH) | TEST<br>TORQUE<br>(FT-LBS) |
|------------------------------|-----------------------------------|---|-------------------------------------|-------------------------------|--------------------------------|--|---------------------------------|----------------------------|
| CASE 1<br>(STRUT P'S)        | % KB−TZ2                          | 1%                                      | 1½) R                               |                               | 31/4                           | 6                                      | SEE <u>PG 28</u>                | 30                         |
| CASE 2                       | ½ KB-TZ2                          | 21/2                                    | 2                                   | 23/4                          | 31/4                           | 8                                      | 3                               | 40                         |
| CASE 3                       | ½ KB−TZ2                          | 21/2                                    | 2                                   | 23/4                          | 4                              | 7 8                                    | 3                               | 40                         |

- THREE (3) CASES OF ATTACHMENT ARE SPECIFIED & PRESENTED IN THIS PRE-APPROVAL:
  - CASE 1: ATTACHMENT DETAILS LOCATED AT UPPER FLRS ABV THE BASE OF A BLDG. THE FLRS ARE ASSUMED TO BE BUILT OF A MIN 31/4" SLWC TOPPING OVER 3" DEEP MIN 20 GA MTL DECK (f'c = 3000 PSI, MIN). 24 ANCHORS SHALL BE CARBON STEEL THRD ROD THRU CONC & MTL DECK. STRUT P EXPANSION ANCHORS SHALL BE 304 OR 316 SS INTO DECK FLUTES.

0 **BLDG ELEV** 

z/h

**ROOF** 

3RD FLR

2ND FLR

**BASE** 

**BASEMENT** 

1

2/3

1/3

ATTACHMENT DETAILS LOCATED AT UPPER FLRS ABV CASE 2: THE BASE OF A BLDG. THE FLRS ARE ASSUMED TO BE BUILT OF A MIN 31/4" SLWC TOPPING OVER 3" DEEP MIN 20 GA MTL DECK (f'c = 3000 PSI, MIN). EXPANSION ANCHORS SHALL BE 304 OR 316 SS & INTO CONC OVER MTL DECK.

CASE 3: ATTACHMENT DETAILS LOCATED AT OR BLW THE BASE OF A BLDG. THE FLRS ARE ASSUMED TO BE BUILT OF A MIN 4" NWC SLAB (f'c = 3000 PSI, MIN). EXPANSION ANCHORS SHALL BE 304 OR 316 SS INTO CONC SLAB.



SHEET TITLE: GENERAL NOTES (CONTINUED)



:\Users\camachom\appdata\loca\\temp\AcPublish\_95368\S1.dwq Time:Jul11,2024-03:40pm Login:camachom Dimscale:12 LTScale:6

CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

<sub>(916) 920-2020</sub>|Date: TEL

23050 Job No: 06-10-2024

www.cyseng.com | Page: 3 of 33



| Δ | R | R | R | F١ | /I | Δ | П | n | N  | S: |  |
|---|---|---|---|----|----|---|---|---|----|----|--|
| ~ |   |   |   |    |    | ~ |   |   | ıv |    |  |

AT

| <b>W</b> | Al                                  | NO. (#) | NOWREK OK POONDS             |
|----------|-------------------------------------|---------|------------------------------|
| AB       | ANCHOR BOLT                         | NWC ``' | NORMAL WEIGHT CONCRETE       |
| ABV      | ABOVE                               | OP      | OPERATING                    |
| ADJ      | ADJACENT                            | OPG     | OPENING                      |
| ASCE     | AMERICAN SOCIETY OF CIVIL ENGINEERS | OPM     | HCAI PRE-APPROVAL OF         |
| BLDG     | BUILDING                            |         | MANUFACTURER'S CERTIFICATION |
| BLW      | BELOW                               | PERP    | PERPENDICULAR                |
| BOTT     | BOTTOM                              | PG      | PAGE                         |
| BYD      | BEYOND                              | P_      | PLATE                        |
| CBC      | CALIFORNIA BUILDING CODE            | PSI     | POUNDS PER SQUARE INCH       |
| CG       | CENTER OF GRAVITY                   | REO.    | RECHIRED                     |

KŁQ **CENTERLINE** STRUCTURAL ENGINEER OF RECORD **SEOR** CONC CONCRETE

SIM **SIMILAR** CONN CONNECTION **SLWC** SAND-LIGHTWEIGHT CONCRETE

COORD COORDINATE **SPCG SPACING DOUBLE** DBL STAINLESS STEEL SS

DETAIL(S) DTL(S) STL STEEL DIA (Ø) DIAMETER THK THICK/THICKNESS

**EXISTING CONDITION** (E) ANCHORAGE TENSION REACTION DUE TO Tu ĒΑ **EACH** 

SEISMIC FORCE AT LRFD ΕE EACH END THRD THREAD OR THREADED **ELEV ELEVATION** TOP & BOTTOM T&B FQUAL TYPICAL FΩ 5.5YP

**EQUIP EQUIPMENT** UNO UNLESS NOTED OTHERWISE

f'c MINIMUM ULTIMATE COMPRESSIVE ANCHORAGE SHEAR REACTION DUE TO Vu

STRENGTH OF CONCRETE BY SEISMIC FORCE AT LRFD **FLOOR** W/ WITH

FOOT/FEET Wp FT (') COMPONENT OPERATING WEIGHT SPECIFIED YIELD STRENGTH OF WEIGHT Fy

REINFORCING, PS OF STEEL, KSI GA **GAUGE** 

HEALTH CARE ACCESS & INFORMATION **HCAI** 

HOLLOW STRUCTURAL SECTION HSS INTERNATIONAL CODE COUNCIL ICC

IN (") INCH **INFO** INFORMATION

KSI KIPS PER SQUARE INCH

**LBS POUNDS** 

**LRFD** LOAD & RESISTANCE FACTOR DESIGN

MAX **MAXIMUM** MIN **MINIMUM MILLIMETER** mm **MODULE** MOD MTL **METAL** 

ATE OF CALIF

SHEET TITLE: ABBREVIATIONS



CYS STRUCTURAL ENGINEERS, INC.

TEL 2710 GATEWAY OAKS DRIVE, SUITE 190N

www.cyseng.com Page: SACRAMENTO, CA 95833

(916) 920-2020 Date:

23050 Job No: 06-10-2024 4 of 33

FLR



#### **DESIGN CRITERIA:**

SUPPORT & ATTACHMENT DESIGN IS PER 2022 CBC AT LRFD LEVEL FORCES.

OTHER MECHANICAL OR ELECTRICAL COMPONENTS PER TABLE 13.6-1 OF ASCE 7-16 INCL SUPPLEMENT #1 & ERRATA:

$$a_{D} = 1.0$$

$$R_{\rm p} = 1.5$$

$$I_0 = 1.5$$

$$I_0 = 1.5$$
  $\Omega_0 = 1.5$  (FOR CONC ANCHORS ONLY)

WP AS NOTED ON DRAWINGS

UPPER FLRS ABV THE BASE OF BLDG

$$S_{DS} \le 1.40$$
  
 $S_{DS} < 0.80$ 

$$\begin{array}{lll} F_p = 1.12 \ W_p & z/h \leq 0.50 \\ F_p = 0.80 \ W_p & z/h \leq 0.75 \end{array}$$

$$z/h \leq 0.50$$

FLRS AT OR BLW THE BASE OF BLDG

$$S_{DS} \leq 2.50$$

$$F_{p} = 1.13 W_{p} z/h = 0$$

$$z/h = 0$$

LOAD COMBINATIONS

$$(0.9 - 0.2 S_{DS}) D - \Omega_0 F_0$$
 (FOR MAX TENSION)

$$(0.9 - 0.2 S_{DS})$$
 D  $- \Omega_0$  F<sub>p</sub> (FOR MAX TENSION)  $(1.2 + 0.2 S_{DS})$  D  $+ \Omega_0$  F<sub>p</sub> (FOR MAX COMPRESSION)

THIS PRE-APPROVAL MAY BE USED ONLY AT GEOGRAPHICAL LOCATIONS IN THE STATE OF CALIFORNIA WHERE SDS & Z/h IS LESS THAN OR EQ TO THE VALUES NOTED ABV. SEOR SHALL VERIFY THAT OTHER COMBINATIONS OF SDS & Z/h MUST RESULT IN AN FP VALUE THAT IS EQ TO OR LESS THAN Fp FORCE FOR CASE UNDER CONSIDERATION.

**BY** Timothy Piland



SHEET TITLE: DESIGN CRITERIA



:\Users\camachom\appdata\loca\\temp\AcPublish\_95368\S1.dwq Time:Jul11,2024-03:40pm Login:camachom Dimscale:12 LTScale:6


CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N TEL (916) 920-2020 Date: SACRAMENTO, CA 95833

Job No: 23050 06-10-2024

www.cyseng.com Page: 5 of 33



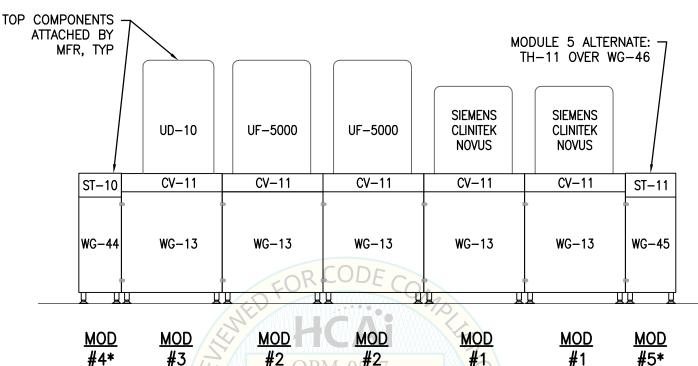

CONFIGURATION #9 - ELEV

BY: Timothy Piland

**NOTES:** 

- SEE PG 8 FOR TABLE LISTING CONFIGURATIONS.
- MODULES W/ AN "\*" INDICATES THAT THESE ARE OPTIONAL. FOR MODULE

INTERCONNECTIONS, SEE PG 6.



SHEET TITLE: CONFIGURATION EXAMPLES



CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024

www.cyseng.com Page: 7 of 33



|                     |              |                                     | CONFI                              | GURATIO                            | N MATR                             | RIX              | ,               |       |    | IES:   |
|---------------------|--------------|-------------------------------------|------------------------------------|------------------------------------|------------------------------------|------------------|-----------------|-------|----|--|
|                     | MODULE #     | 1                                   |                                    |                                    |                                    |                  |                 |       | 1. | MODULES W/ AN "*"  |
| CONFIGURATION       | COMPONENT(S) | SIEMENS CLINITEK<br>NOVUS           |                                    |                                    |                                    |                  |                 |       | 4  | INDICATES THAT THESE   |
| 1                   | WAGON        | CV-11<br>WG-13                      |                                    |                                    |                                    |                  |                 |       | 1  | ARE OPTIONAL IN THE  |
|                     | MODULE #     | 2 WG-13                             |                                    |                                    |                                    |                  |                 |       | 1  | CONFIGURATION,   |
| CONFIGURATION 2     | COMPONENT(S) | UF-5000                             |                                    |                                    |                                    |                  |                 |       | 2. | THE ABV CONFIGURATION MATRIX REPRESENTS KEY  |
| 2                   | WAGON        | CV-11<br>WG-13                      |                                    |                                    |                                    |                  |                 |       | 1  | EXAMPLES OF UN-SERIES  |
|                     | MODULE #     | 3                                   |                                    |                                    |                                    |                  |                 |       | 1  | CONFIGURATIONS. THERE  |
| CONFIGURATION 3     | COMPONENT(S) | UD-10<br>CV-11                      |                                    |                                    |                                    |                  |                 |       | 1  | MAY BE OTHER   |
|                     | WAGON        | WG-13                               |                                    |                                    |                                    |                  |                 |       | 1  | CONFIGURATIONS THAT  |
|                     | MODULE #     | 2                                   | 1                                  |                                    |                                    |                  |                 |       | 1  | SYSMEX DESIGNS USING   |
| CONFIGURATION 4     | COMPONENT(S) | UF-5000<br>CV-11                    | SIEMENS CLINITEI<br>NOVUS<br>CV-11 | (                                  |                                    |                  |                 |       | 1  | THE ABV MODULES FOR SITE SPECIFIC PROJECTS   |
|                     | WAGON        | WG-13                               | WG-13                              |                                    |                                    |                  |                 |       |    | SIL SI LOI IC I NOOLOIS  |
|                     | MODULE #     | 4*                                  | 2                                  | 1                                  | 5*                                 |                  |                 |       |    |  |
| CONFIGURATION 5     | COMPONENT(S) | ST-10                               | UF-5000<br>CV-11                   | SIEMENS CLINITER<br>NOVUS<br>CV-11 | ST-11                              |                  |                 |       |    |  |
|                     | WAGON        | WG-44                               | WG-13                              | WG-13                              | WG-45                              | 1- (             |                 |       |    |  |
|                     | MODULE #     | 2                                   | 1                                  | 6*                                 | TXXXXXXXXX                         | XXXXX            | ***             |       |    |  |
| CONFIGURATION<br>6  | COMPONENT(S) | UF-5000<br>CV-11                    | SIEMENS CLINITEI<br>NOVUS<br>CV-11 | ST-12                              | Wanti                              |                  |                 |       |    |  |
|                     | WAGON        | WG-13                               | WG-13                              | WG-45                              | Y AMMA                             | <u> </u>         |                 | 7     |    |  |
|                     | MODULE #     | 4*                                  | 3                                  | 2                                  | -X/Av-1Xv/                         | 5*               |                 | 11    |    | MODULE 5 ALTERNATE:  |
| CONFIGURATION 7     | COMPONENT(S) | ST-10                               | UD-10<br>CV-11                     | UF-5000<br>CV-11                   | SIEMENS CLINITEI<br>NOVUS<br>CV—11 | ST-11            |                 |       | /  | TH-11 OVER WG-46   |
|                     | WAGON        | WG-44                               | WG-13                              | WG-13                              | WG-13                              | WG-45            |                 |       | Y  |  |
|                     | MODULE #     | 3                                   | 2                                  | WXXXX XXXXX                        | 6*                                 |                  |                 |       | 1  |  |
| CONFIGURATION       | COMPONENT(S) | UD-10                               | UF-5000                            | SIEMENS CLINITER<br>NOVUS          | ST-12P                             | iland            |                 | //    |    |  |
| 8                   | WAGON        | CV-11<br>WG-13                      | CV-11<br>WG-13                     | CV-11<br>WG-13                     | WG-45                              |                  |                 |       | -  |  |
|                     | MODULE #     | 4*                                  | 3                                  | 2                                  | 2                                  | 1                | 1 /             | 5*    | 1  |  |
| CONFIGURATION       |              | ST-10                               | UD-10                              | UF-5000                            | UF-5000                            | SIEMENS CLINITER | SIEMENS CUNITER | ST-11 | 1  |  |
| 9                   | COMPONENT(S) |                                     | CV-11                              | CV-11                              | CV-11                              | NOVUS<br>CV-11   | CV-11           | 707/  | 1  |  |
|                     | WAGON        | WG-44                               | WG-13                              | WG-13                              | WG-13                              | WG-13            | <b>W</b> G−13   | WG-45 | 1  |  |
|                     | MODULE #     | 3                                   | 2                                  | 2                                  | 1                                  | MAN              | 6*              | 7/    |    |  |
| CONFIGURATION<br>10 | COMPONENT(S) | UD-10<br>CV-11                      | UF-5000<br>CV-11                   | UF-5000<br>CV-11                   | NOVUS<br>CV-11                     | NOVUS<br>CV-1/1  | ST-12           | ///   |    |  |
|                     | WAGON        | VAGON WG-13 WG-13 WG-13 WG-13 WG-45 |                                    |                                    |                                    |                  |                 |       |    |  |
|                     | MODULE #     | 3                                   | 2                                  | 11                                 |                                    |                  |                 |       |    |  |
| CONFIGURATION<br>11 | COMPONENT(S) | UD-10<br>CV-11                      | UF-5000<br>CV-11                   | JBI                                | <b>JILD</b>                        | MO               |                 |       |    |  |
|                     | WAGON        | WG-13                               | WG-13                              |                                    |                                    |                  |                 |       |    | The state of the s |
|                     | MODULE #     | 4*                                  | 3                                  | 2                                  | 5*′                                |                  |                 |       |    | PROFESS /ON TOWN   |
| CONFIGURATION<br>12 | COMPONENT(S) | ST-10                               | UD-10<br>CV-11                     | UF-5000<br>CV-11                   | ST-11                              |                  |                 |       |    | PROFESS /ONA/  |
|                     | WAGON        | WG-44                               | WG-13                              | WG-13                              | WG-45                              |                  |                 |       |    | 85.4   |
|                     | MODULE #     | 3                                   | 2                                  | 6*                                 |                                    |                  |                 |       |    | SHOW THE STATE OF  |
| CONFIGURATION<br>13 | COMPONENT(S) | UD-10<br>CV-11                      | UF-5000<br>CV-11                   | ST-12                              |                                    |                  |                 |       |    | No. S4346  |
|                     | WAGON        | WG-13                               | WG-13                              | WG-45                              |                                    |                  |                 |       |    | - X   V   -  |

SHEET TITLE: CONFIGURATION TABLE



CYS STRUCTURAL ENGINEERS, INC.


2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833 TEL (916) 920-2020 Date:

www.cyseng.com Page:

Job No: 23050 Date: 06-10-2024


m Page: 8 of 33



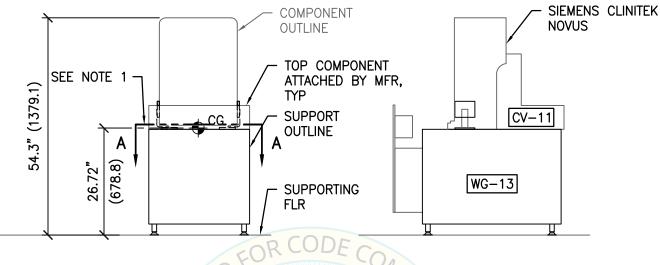

FRONT ELEV

LEFT SIDE ELEV

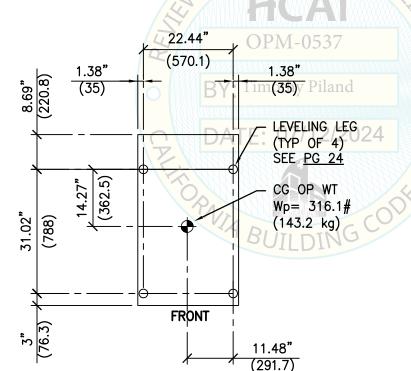

NOTES:

- FOR COMPONENT CONNECTIONS TO SUPPORT, SEE <u>PG 15.</u> DIMS IN PARENTHESES ARE mm
- & kg, TYP THIS OPM.
- CASTERS ARE NOT SHOWN IN THIS OPM FOR CLARITY, TYP



SHEET TITLE: MODULE #1

PLAN VIEW & ELEVATION

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6

CYS STRUCTURAL ENGINEERS, INC.

PLAN AT BASE

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL


23050 Job No: 06-10-2024 9 of 33 www.cyseng.com Page:

#### **UN SERIES SYSTEM** sysmex Lighting the way COMPONENT **OUTLINE** UF-5000 SEE NOTE 1 -TOP COMPONENT 60.79" (1544) ATTACHED BY MFR, <del></del>CG CV-11 **SUPPORT OUTLINE** (822)32.36" WG-13 **SUPPORTING** C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6 FRONT ELEV LEFT SIDE ELEV 22.44" (570.1)1.38" 1.38" (35)i(35) hy Piland LEVELING LEG (TYP OF 4) SEE PG 24 NOTES: CG OP WT FOR COMPONENT CONNECTIONS Wp= 331.2# TO SUPPORT, SEE PG 16. 31.02" (788) (150.2 kg) SEE PG 9 FOR TYPICAL NOTES. **FRONT** 12.82" (325.7) PLAN AT BASE E OF CALIF SHEET TITLE: MODULE #2 PLAN VIEW & ELEVATION CYS STRUCTURAL ENGINEERS, INC. Job No: 23050 (916) 920-2020 Date: 06-10-2024 TEL 2710 GATEWAY OAKS DRIVE, SUITE 190N www.cyseng.com Page: 10 of 33

SACRAMENTO, CA 95833

#### **UN SERIES SYSTEM** sysmex Lighting the way COMPONENT OUTLINE UD-10 TOP COMPONENT 60.79" (1544) SEE NOTE 1 · ATTACHED BY MFR. 🕁 cg **SUPPORT** CV-11 **OUTLINE** (762.4)30.02" WG-13 **SUPPORTING** FLR C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6 FRONT ELEV LEFT SIDE ELEV 22.44" (570)1.38" 1.38" (35.1)thy Piland 8.69" (35.1)LEVELING LEG (TYP OF 4) SEE PG 24 **NOTES:** CG OP WT FOR COMPONENT CONNECTIONS 31.02" (788) Wp= 298.1# TO SUPPORT, SEE PG 17. (135.2 kg) SEE PG 9 FOR TYPICAL NOTES. FRONT 3.07 10.04" (254.9)PLAN AT BASE E OF CALIF SHEET TITLE: MODULE #3 **PLAN VIEW & ELEVATION** CYS STRUCTURAL ENGINEERS, INC. Job No: 23050 (916) 920-2020 Date: 06-10-2024 TEL 2710 GATEWAY OAKS DRIVE, SUITE 190N www.cyseng.com Page: 11 of 33 SACRAMENTO, CA 95833



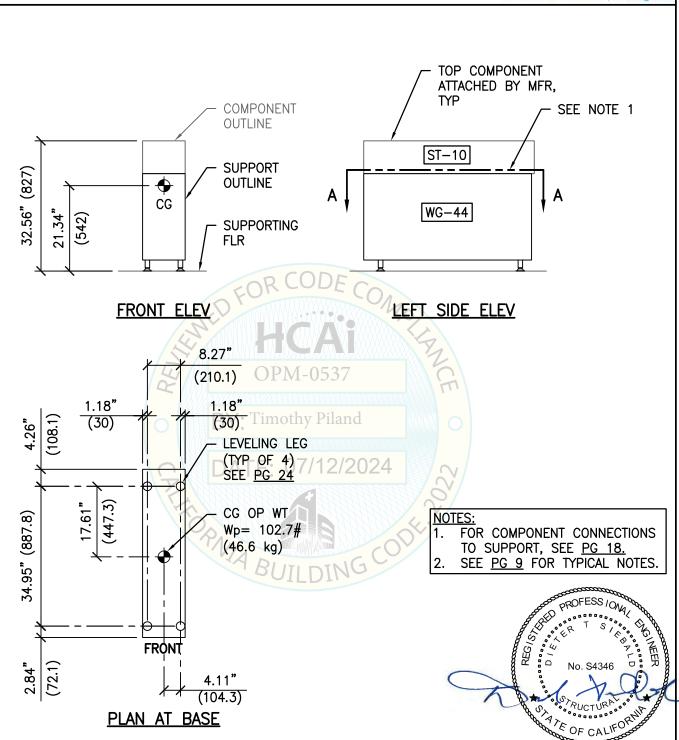

SHEET TITLE: MODULE #4

**PLAN VIEW & ELEVATION** 

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6


CYS STRUCTURAL ENGINEERS, INC. 2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

www.cyseng.com Page:

Job No: 23050 06-10-2024 12 of 33



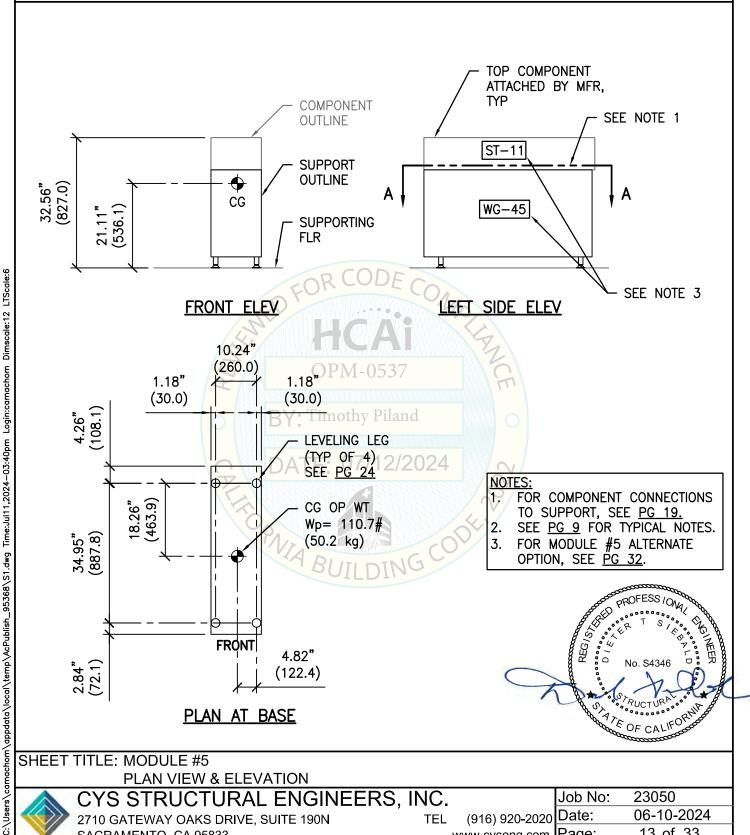

SHEET TITLE: MODULE #5

PLAN VIEW & ELEVATION

CYS STRUCTURAL ENGINEERS, INC. 2710 GATEWAY OAKS DRIVE, SUITE 190N

PLAN AT BASE

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024


SACRAMENTO, CA 95833

www.cyseng.com Page:

13 of 33

OF CALIF



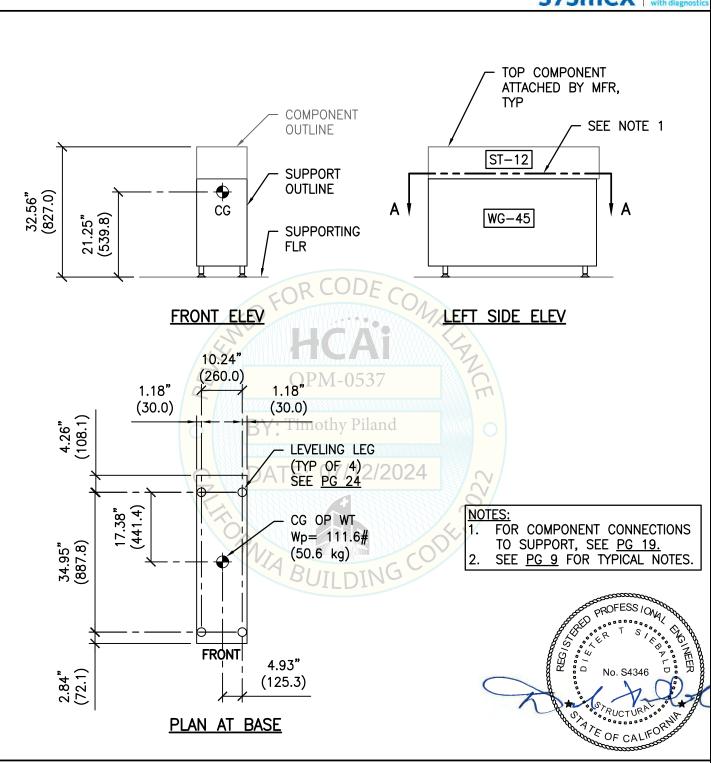

SHEET TITLE: MODULE #6

PLAN VIEW & ELEVATION

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6


CYS STRUCTURAL ENGINEERS, INC. 2710 GATEWAY OAKS DRIVE, SUITE 190N

SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 www.cyseng.com Page: 14 of 33



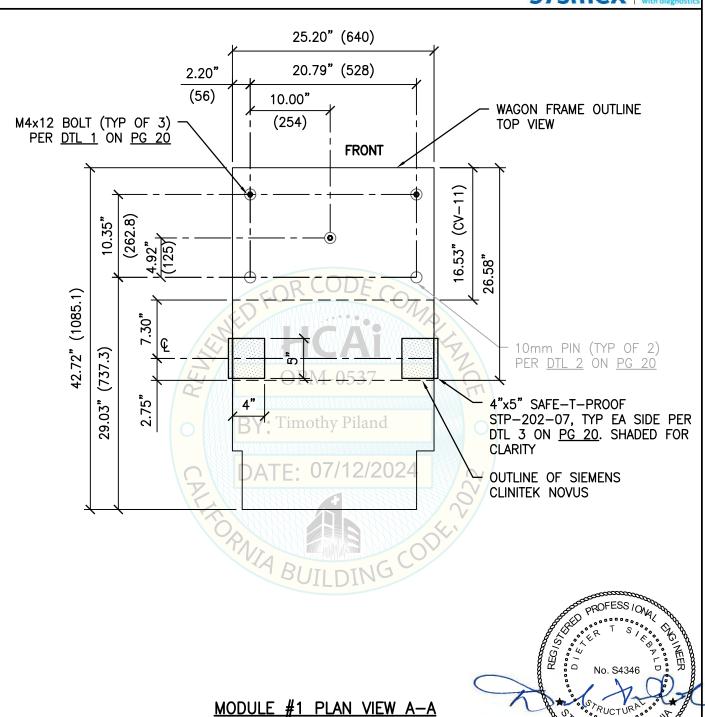

SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT



C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:40pm Login:camachom Dimscale:12 LTScale:6

CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833


(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 15 of 33 www.cyseng.com | Page:

ATE OF CALIF

SIEMENS CLINITEK NOVUS, CV-11, WG-13



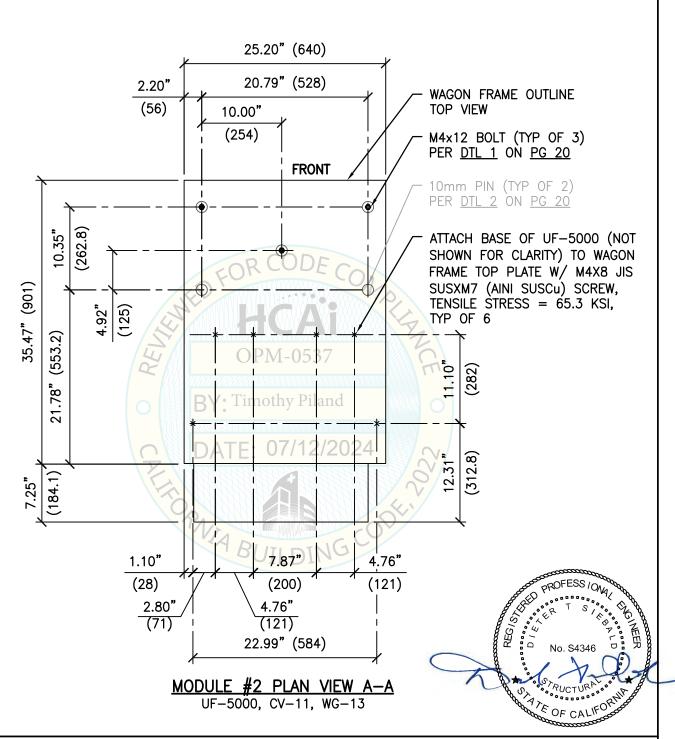

SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT




CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 www.cyseng.com Page: 16 of 33



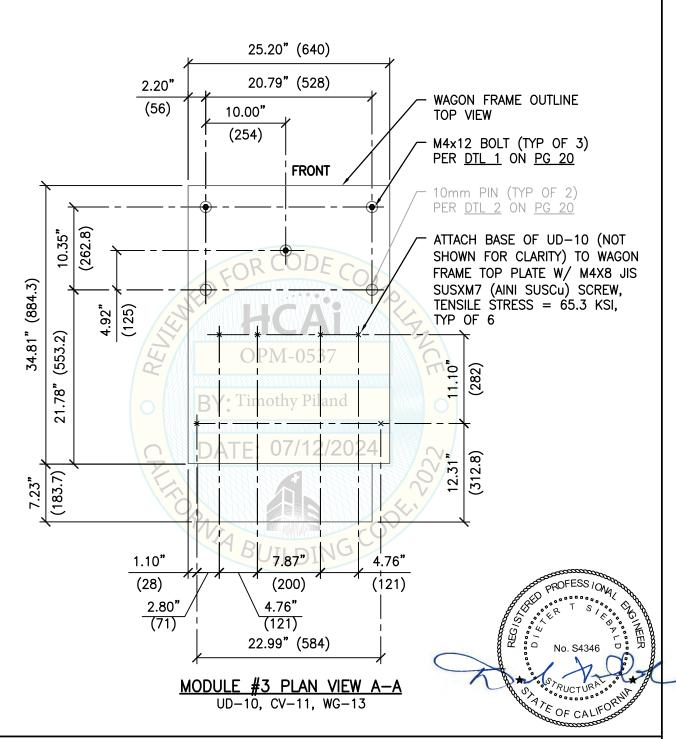

SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT



CYS STRUCTURAL ENGINEERS, INC.


2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 17 of 33

www.cyseng.com Page:



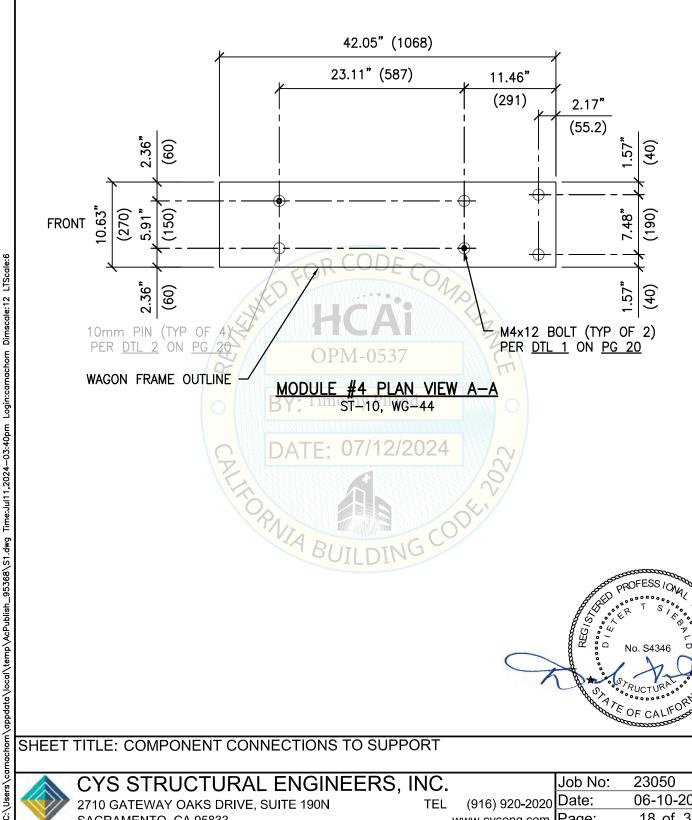



SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT




CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 www.cyseng.com Page: 18 of 33



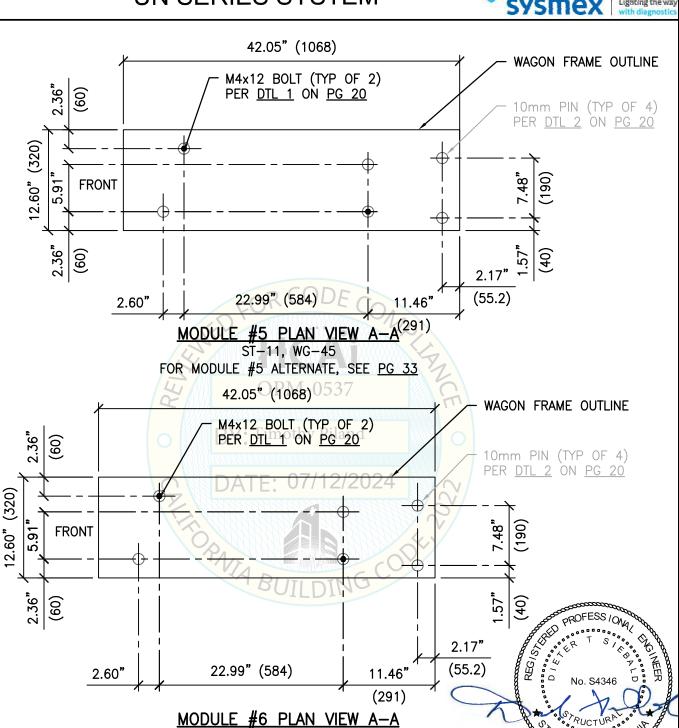

SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT



CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 19 of 33 www.cyseng.com Page:


ATE OF CALIF


07/12/2024


ST-12, WG-45

SACRAMENTO, CA 95833


www.cyseng.com Page:

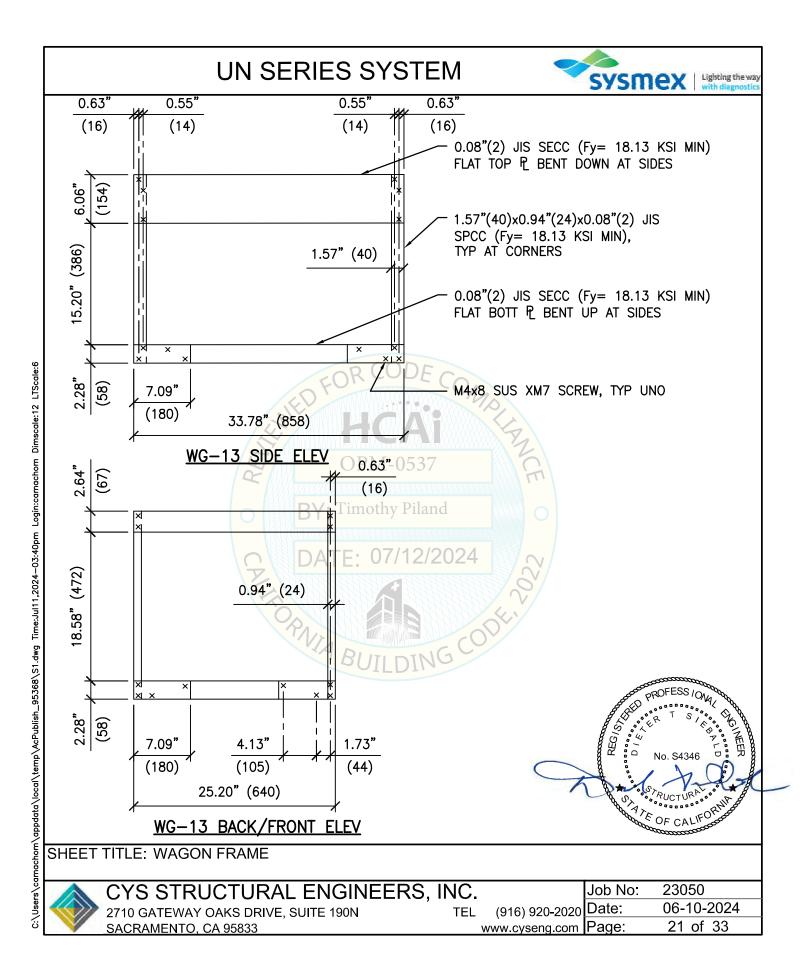

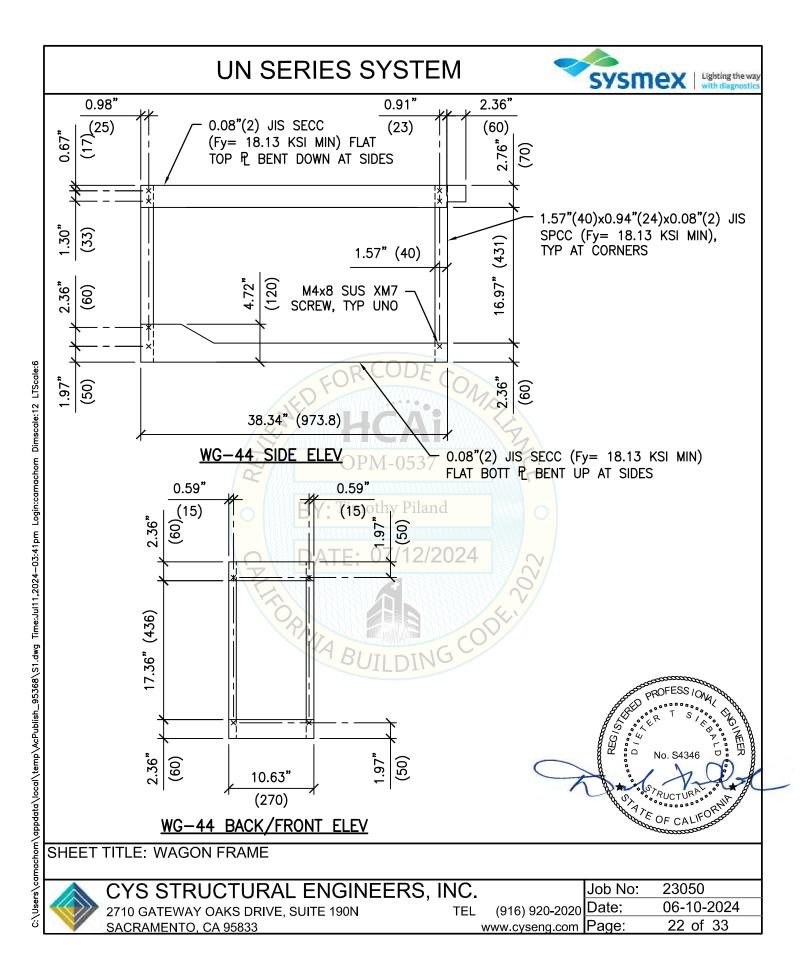

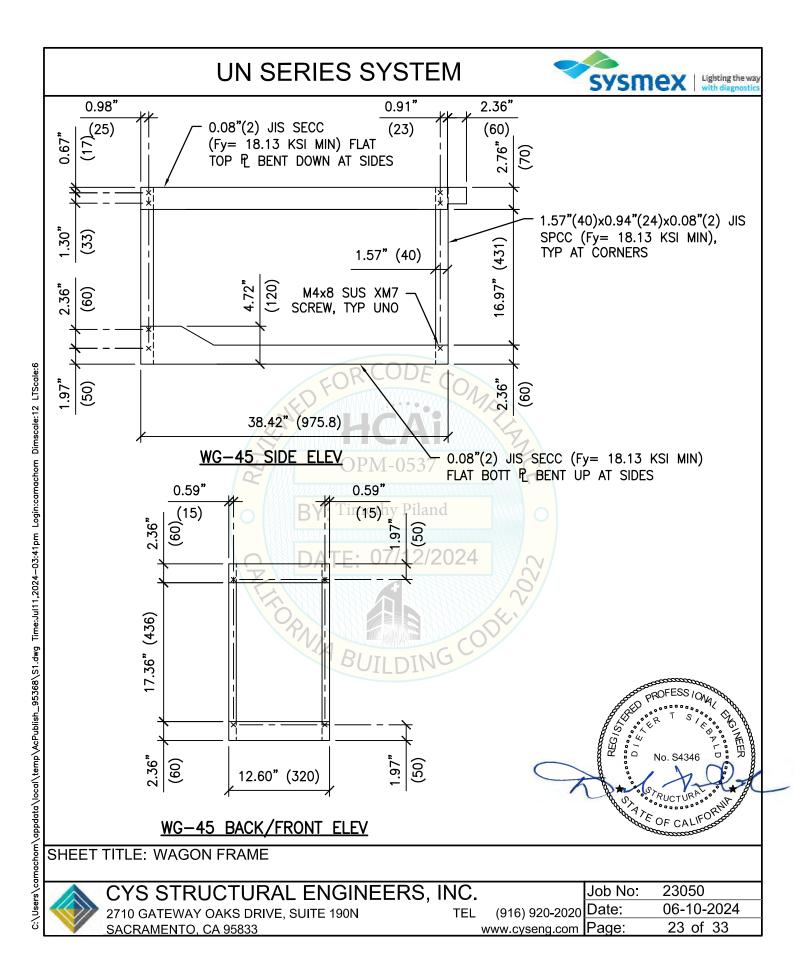



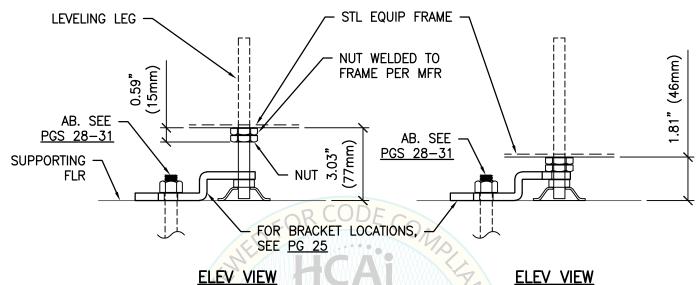

MAX CONDITION

MIN CONDITION

BY: Timothy Piland

DATE: 07/12/2024

M12 JIS SUS304 (ASTM S276 304) STL FULLY THRD LEVELING LEG, Fy= 30 KSI

0.87" (22mm) NUT SUPPLIED BY LEVELING LEG MFR 2.17"ø



SHEET TITLE: LEVELING LEG DETAILS

CYS STRUCTURAL ENGINEERS, INC.

(55mm)

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL


Job No: 23050 06-10-2024 www.cyseng.com Page: 24 of 33

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:41pm Login:camachom Dimscale:12 LTScale:6

# **UN SERIES SYSTEM** sysmex Lighting the way CORNER BRACKET CORNER BRACKET PER PG 26 PER <u>PG 26</u> (TYP OF 4) (TYP OF 4) INTERMEDIATE BRACKET C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:41pm Login:camachom Dimscale:12 LTScale:6 PER <u>PG 27</u> (TYP OF 2) 2 MODULE 1 MODULE CONFIGURATION CONFIGURATION INTERMEDIATE BRACKET PER PG 27, TYP CORNER BRACKET PER PG 26, TYP · Timothy Piland 3 OR MORE MODULE CONFIGURATION SHEET TITLE: BRACKET LOCATIONS

CYS STRUCTURAL ENGINEERS, INC. Job No: 23050 (916) 920-2020 Date: 06-10-2024 TEL 2710 GATEWAY OAKS DRIVE, SUITE 190N www.cyseng.com Page: 25 of 33 SACRAMENTO, CA 95833



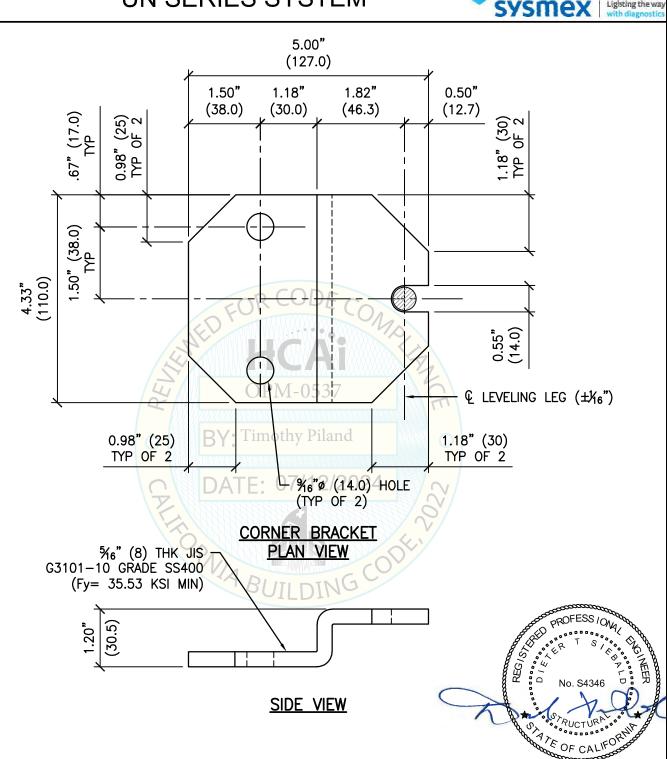

SHEET TITLE: BRACKET DETAILS

CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N

SACRAMENTO, CA 95833

TEL (916) 920-2020 Date:


www.cyseng.com Page:

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:41pm Login:camachom Dimscale:12 LTScale:6


23050

06-10-2024

26 of 33

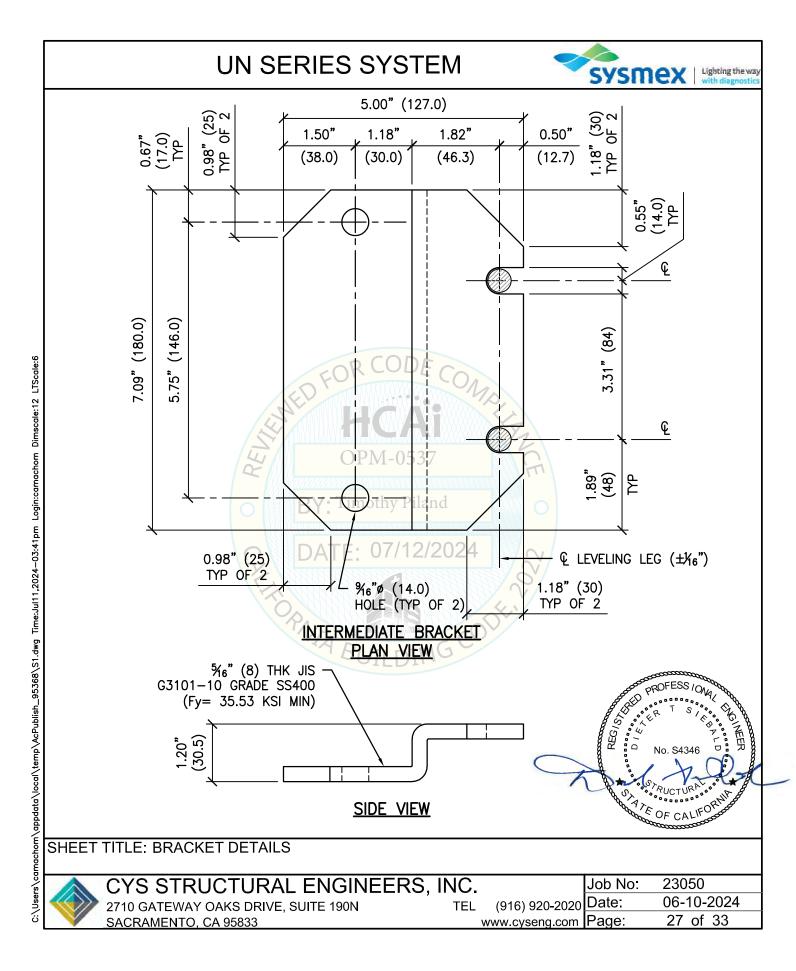



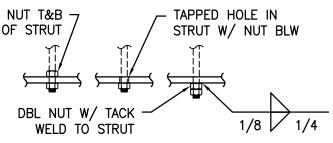



MAX ANCHOR FORCES AT LRFD AT EA AB (LBS) Tu Vu 704# 227# z/h<u><</u>0.75

> $(\Omega_0 = 1.5)$  OVERSTRENGTH FACTOR IS APPLIED TO SHEAR FORCE ONLY

CASE 1

#### **ANCHOR OPTIONS**

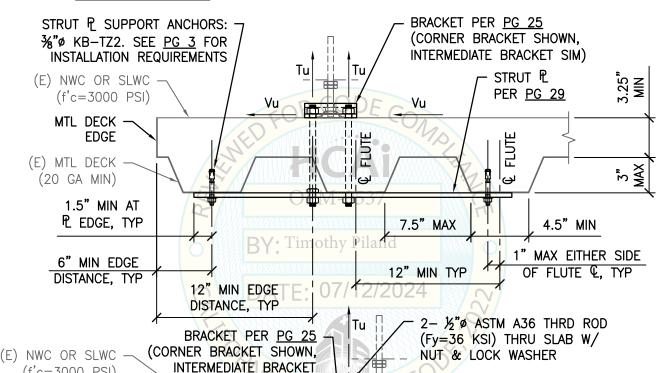

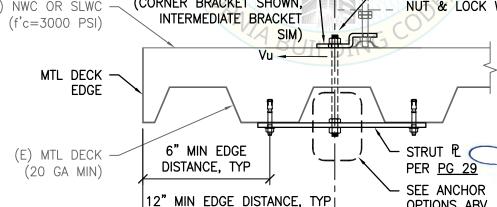

ATE OF CALIF OPTIONS ABV, TYP

SHEET TITLE: ATTACHMENT DETAILS THRU

CONCRETE FILL OVER METAL DECK (CASE 1)



Time:Jul11,2024-03:41pm Login:camachom Dimscale:12 LTScale:6

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwq


CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833

(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 28 of 33 www.cyseng.com Page:



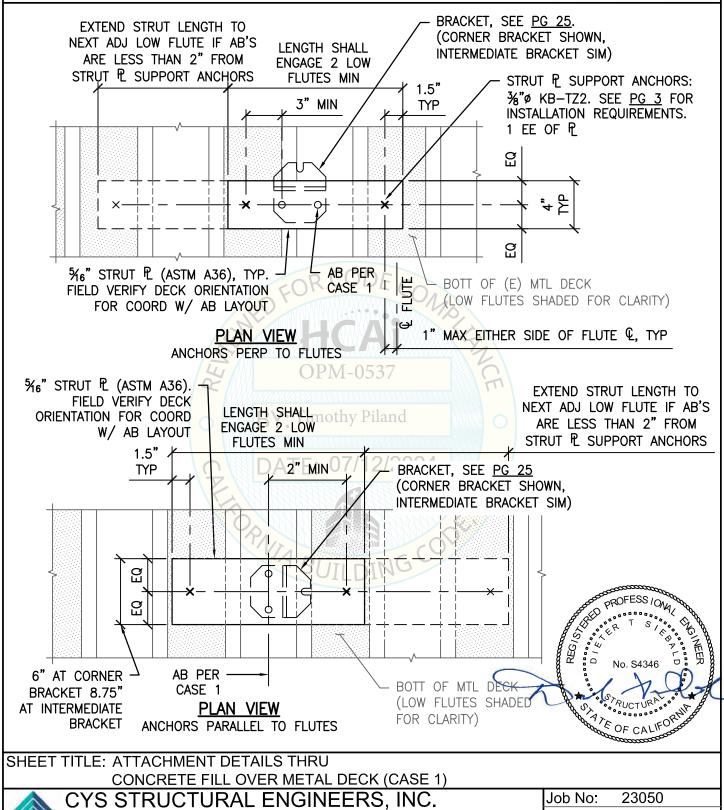

2710 GATEWAY OAKS DRIVE, SUITE 190N

SACRAMENTO, CA 95833

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:41pm Login:camachom Dimscale:12 LTScale:6

TEL

06-10-2024

29 of 33


(916) 920-2020 Date:

www.cyseng.com Page:




MAX ANCHOR FORCES AT LRFD AT EA AB (LBS) Tu Vu CASE 2 688# 162# z/h<u><</u>0.75

INCLUDES OVERSTRENGTH FACTOR ( $\Omega_0$ =1.5)

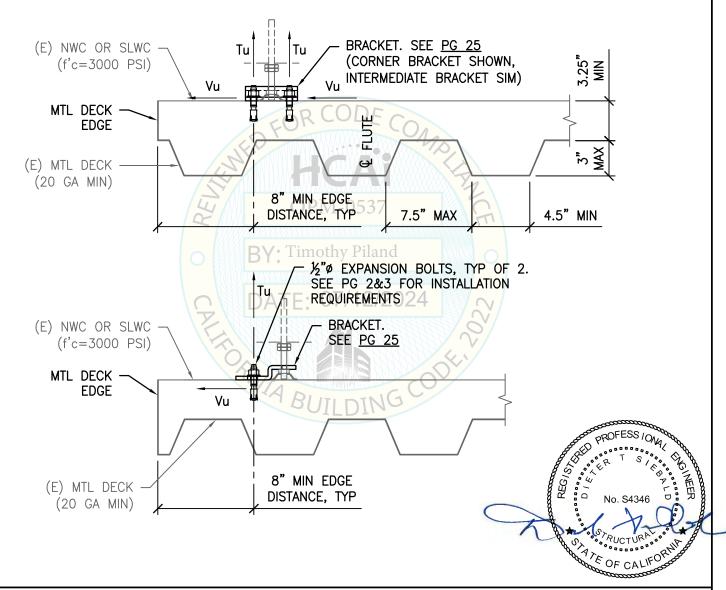

SHEET TITLE: ATTACHMENT DETAILS INTO

CONCRETE FILL OVER METAL DECK (CASE 2)

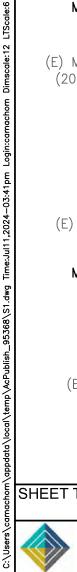

CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833


(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024 30 of 33 www.cyseng.com Page:



MAX ANCHOR FORCES AT LRFD AT EA AB (LBS) Tu ۷u CASE 3 1173# 228# z/h<u><</u>0

INCLUDES OVERSTRENGTH FACTOR ( $\Omega_0$ =1.5)

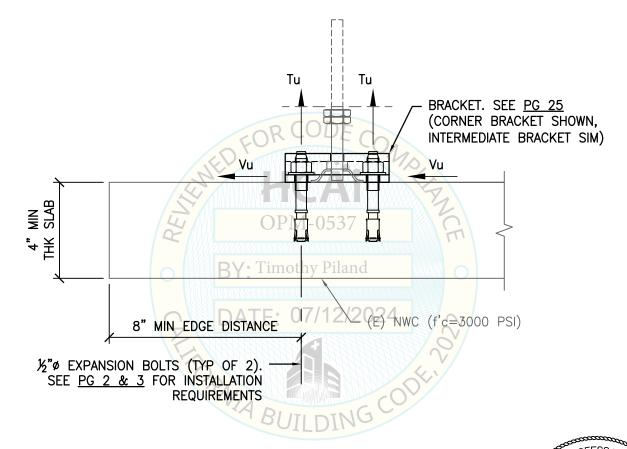

SHEET TITLE: ATTACHMENT DETAILS TO

4" CONCRETE SLAB ON GRADE (CASE 3)

C:\Users\camachom\appdata\local\temp\AcPublish\_95368\S1.dwg Time:Jul11,2024—03:41pm Login:camachom Dimscale:12 LTScale:6

CYS STRUCTURAL ENGINEERS, INC. 2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833


(916) 920-2020 Date: TEL

Job No: 23050 06-10-2024

www.cyseng.com Page:

31 of 33



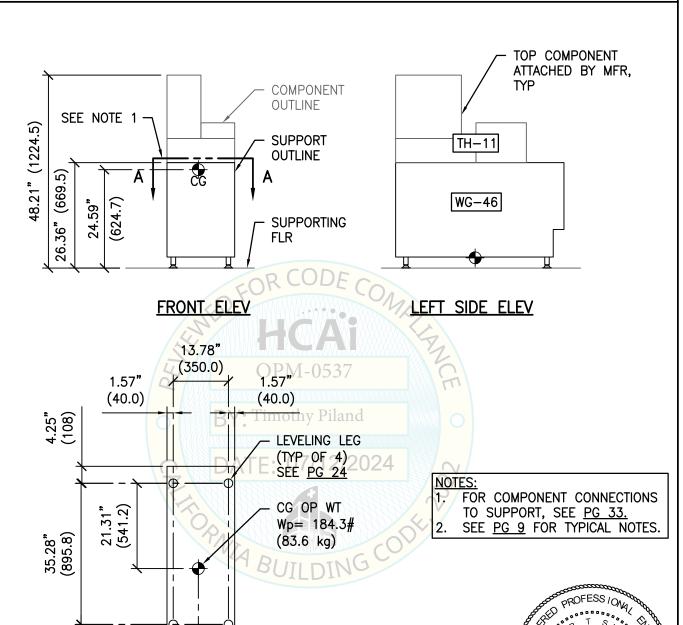

SHEET TITLE: MODULE #5 ALTERNATE OPTION PLAN VIEW & ELEVATION

CYS STRUCTURAL ENGINEERS, INC.

PLAN AT BASE

FRONT

7.50"

(190.6)

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833 TEL (916) 920-2020 Date: www.cyseng.com Page:


Job No: 23050 Date: 06-10-2024 Page: 32 of 33

OF CALIF

2.84" (65.1)



Lighting the way

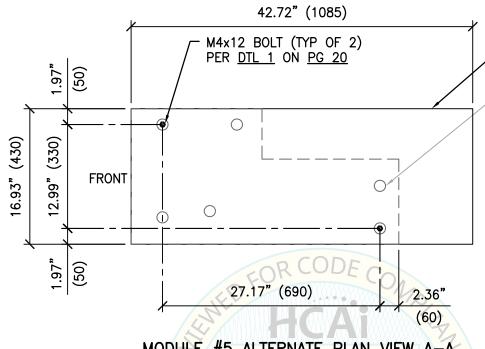

WAGON FRAME OUTLINE

10mm PIN (TYP OF 4) PER DTL 2 ON PG 20

MODULE #5 ALTERNATE PLAN VIEW A-A
TH-11, WG-46

BY: Timothy Piland

DATE: 07/12/2024



SHEET TITLE: COMPONENT CONNECTIONS TO SUPPORT



CYS STRUCTURAL ENGINEERS, INC.

2710 GATEWAY OAKS DRIVE, SUITE 190N SACRAMENTO, CA 95833 TEL (916) 920-2020 Date: www.cyseng.com Page:

 Job No:
 23050

 Date:
 06-10-2024

 Page:
 33 of 33

07/12/2024

OPM-0537: Reviewed for Code Compliance by Timothy Piland