

OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION

allital i lat.			
APPLICATION FOR OSHPD PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM)		OFFICE USE ONLY	
		APPLICATION #: OPM-0592	
OSHPD Preapproval of Manufacturer's Certification (OPM)			
Type: X New Renewal/Update			
Manufacturer Information			
Manufacturer: Carestream Health			
Manufacturer's Technical Representative: Christopher Kralles			
Mailing Address: 1049 Ridge Road West, Rochester, NY 14615			
Telephone: (800) 328-2910 Email: christopher.kralles@carestreamhealth.com			
	FOR CODE CO.		
Product Information OSHPD			
Product Name: DRX-Excel Plus 480/240 Tra	ansformer (for PDU)	Z	
Product Type: Transformer	OPM-0592	CE	
Product Model Number: N/A	BY: Jeffrey Kikumoto		
General Description: Power Supply for PDU	J		
A	DATE: 12/02/2021	202	
		N	
Applicant Information			
Applicant Company Name: EASE LLC.			
Contact Person: Tiffany Tonn	BUILDING		

"Access to Safe. Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"

Mailing Address: 1515 FAIRVIEW AVE, STE 205, MISSOULA, MT 59801

STATE OF CALIFORNIA – HEALTH AND HUMAN SERVICES AGENCY





Telephone: (406) 541-3273

Title:

Email: tiffany@easeco.com



OFFICE OF STATEWIDE HEALTH PLANNING AND DEVELOPMENT FACILITIES DEVELOPMENT DIVISION

Registered Design Professonal Preparing Engineering Recommendations		
Company Name: EASE LLC		
Name: Kevin Paul Burke California License Number: CE57152		
Mailing Address: 5877 Pine Ave., Suite 210, Chino Hills, CA 91709		
Telephone: (909) 606-7622 Email: kevin@easeco.com		
OSHPD Special Seismic Certification Preapproval (OSP)		
Special Seismic Certification is preapproved under OSP OSP Number: OSP-0675		
Special Seismic Certification is preapproved under OSF OSF Number. OSF-0073		
Certification Method		
Testing in accordance with: ICC-ES AC156 FM 1950-16		
Other(s) (Please Specify):		
*Use of criteria other than those adopted by the California Building Standards Code, 2019 (CBSC 2019) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2019 may be used when approved by OSHPD prior to testing.		
X Analysis		
Experience Data DATE: 12/02/2021		
Combination of Testing, Analysis, and/or Experience Data (Please Specify):		
CODE		
OSHPD Approval BUILDING		
Date: 12/2/2021		
Name: Jeffrey Kikumoto Title: Senior Structural Engineer		
Condition of Approval (if applicable):		

"Access to Safe. Quality Healthcare Environments that Meet California's Diverse and Dynamic Needs"









EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

Office of Statewide Health Planning and Development
PREAPPROVAL OF MANUFACTURER'S CERTIFICATION

OPM-0592

THIS PREAPPROVAL CONFORMS TO THE 2019 CALIFORNIA BUILDING CODE

MANUFACTURER: EQUIPMENT NAME:

CARESTREAM HEALTH, INC

DRX EXCEL PLUS DIGITAL R/F SYSTEM 480/240 TRANSFORMER FOR PDU

Sheet: <u>1 of 3</u>

Date: 9/3/21

GENERAL NOTES

- 1. THIS OSHPD PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2019 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2019 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2019 CALIFORNIA BUILDING CODE.
- 4. FORCES PER ASCE 7-16 SECTION 13.3.1, EQUATIONS 13.3-1, 13.3-2 & 13.3-3, WHERE SDS = 2.30, **a**_p = 1.0, I_p = 1.5, R_p = 2.5, z/h < 1.
- 5. THE DETAILS IN THIS PREAPPROVAL MAY BE USED AT ANY LOCATION IN THE STATE OF CALIFORNIA, WHERE SDS IS NOT GREATER THAN 2.30.

 BY: Jeffrey Kikumoto
- 6. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. SHEET METAL SCREWS SHALL BE TEKS SCREWS BY ITW BUILDEX (ICC ESR-1976).
- 8. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- 9. STEEL STUD WALL AND WOOD STUD WALL DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING (I.E. z/h < 1)

10. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE REQUIRED TO SUPPORT WEIGHTS AND FORCES SHOWN, IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2019 CBC AND WITH THE DETAILS SHOWN IN THIS PREAPPROVAL. VERIFY THAT THE ACTUAL EQUIPMENT'S WEIGHT, CG LOCATION, ANCHOR LOCATIONS, ANCHOR DETAILS AND THE MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT THE COMBINATION OF SDS & z/h RESULT IN SEISMIC FORCES (Eh , Ev) THAT ARE NOT GREATER THAN THE VALUES ON THE DETAILS.
- D. DESIGN BACKING BARS, STUDS, ETC. WHICH THE UNITS ARE ATTACHED TO AS NOTED ON THE DRAWINGS.

EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

CARESTREAM HEALTH, INC

DRX EXCEL PLUS DIGITAL R/F SYSTEM 480/240 TRANSFORMER FOR PDU

DES. J. ROBERSON

JOB NO. 11-2002


DATE 9/3/21

SHEET 2

_F 3 sheets

SEISMIC SUPPORTS & ATTACHMENTS

<u>WALL MOUNTED</u>

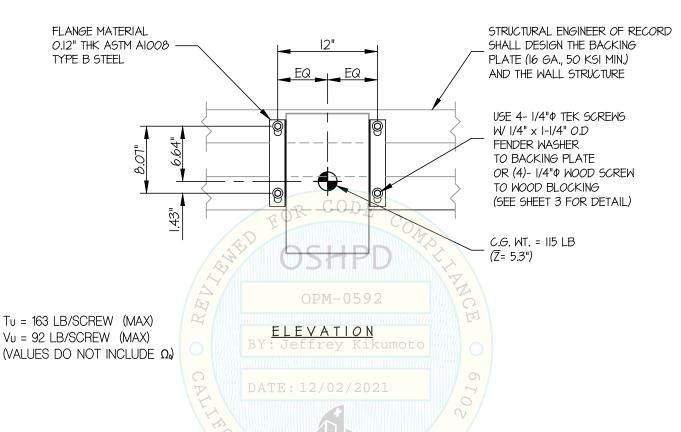

NOTES:

1. FORCES ARE DETERMINED PER 2019 CALIFORNIA BUILDING CODE AND ASCE 7-16

STRENGTH DESIGN IS USED. (SDS = 230 ap = 1.0, Ip = 1.5, Rp = 2.5, $z/h \le 1$)

HORIZONTAL FORCE (En) = 1.66 Wp VERTICAL FORCE (Ev) = 0.46 Wp

- 2. CENTER OF GRAVITY (C.G.) AND WEIGHT ARE THE GOVERNING PARAMETERS FOR DESIGN. THIS PREAPPROVAL ENCOMPASSES ALL WEIGHTS UP TO THE MAXIMUM WEIGHT SHOWN.
- 3. STRUCTURAL ENGINEER OF RECORD FOR THE BUILDING SHALL PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 4. 480/240 TRANSFORMER FOR PDU (UUT-2001-3) HAS OBTAINED SPECIAL SEISMIC CERTIFICATION REFER TO OSP-0675.
- 5. SEE GENERAL NOTES: SHEET 1.



EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING

www.EquipmentAnchorage.com

CARESTREAM HEALTH, INC

DRX EXCEL PLUS DIGITAL R/F SYSTEM 480/240 TRANSFORMER FOR PDU

DES. J. ROBERSON

11-2002 JOB NO.

9/3/21 DATE

(DOUGLAS-FIR LARCH

(DESIGNED BY STRUCTURAL ENGINEER OF RECORD)

4 x BLKG

NUMBER 2 MIN.)

SHEET

OF SHEETS

WALL MOUNTED

3"

MIN

SEISMIC SUPPORTS & ATTACHMENTS

5/8" THK.

II5 LB

WALL BOARD

(I LAYER MAX)

TO BACKING PLATE

STRUCTURAL ENGINEER OF RECORD SHALL DESIGN THE BACKING PLATE (16 GA., 50 KSI MIN.) AND THE WALL STRUCTURE

- Tu C.G. WT. = Vυ ► Tu

USE 4- 1/4" P TEK SCREWS W/ I/4" x I-I/4" O.D FENDER WASHER

Ýυ

NOTE: BY: Jeffrey KikumotoMIN EDGE DISTANCE = I"

USE 4- 1/4" \$\phi \times 4"

WOOD SCREWS TO WOOD BLKG. (PRE-DRILL HOLES

TO 70% SHANK DIAMETER) 5/8" THK.

WALL BOARD (I LAYER MAX)

MIN END DISTANCE = 2"

STEEL STUD WALLSECTION: 12/02/2021

WOOD STUD WALL SECTION

