

### DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION FACILITIES DEVELOPMENT DIVISION

| and his section of the section of th |                         |  |  |  |  |  |  |  |
|--|-------------------------|--|--|--|--|--|--|--|
| APPLICATION FOR HCAI PREAPPROVAL OF  | OFFICE USE ONLY         |  |  |  |  |  |  |  |
| MANUFACTURER'S CERTIFICATION (OPM)   | APPLICATION #: OPM-0702 |  |  |  |  |  |  |  |
| HCAI Preapproval of Manufacturer's Certification (OPM)   |                         |  |  |  |  |  |  |  |
| Type: X New Renewal/Update   |                         |  |  |  |  |  |  |  |
| Manufacturer Information   |                         |  |  |  |  |  |  |  |
| Manufacturer: Werfen   |                         |  |  |  |  |  |  |  |
| Manufacturer's Technical Representative: Steve Leslie  |                         |  |  |  |  |  |  |  |
| Mailing Address: 180 Hartwell Road, Bedford, MA 01730  |                         |  |  |  |  |  |  |  |
| Telephone: (800) 955-9525  | verfen.com              |  |  |  |  |  |  |  |
|  | ONID                    |  |  |  |  |  |  |  |
| Product Information  | T S                     |  |  |  |  |  |  |  |
| Product Name: ACL TOP 770 LAS OPM-0702   |                         |  |  |  |  |  |  |  |
| Product Type: other mechanical or electrical components  |                         |  |  |  |  |  |  |  |
| Product Model Number: ACL TOP 770 LAS BY: William Stael  | alin                    |  |  |  |  |  |  |  |
| General Description: Medical equipment used for hemostasis testing   | 2002                    |  |  |  |  |  |  |  |
| PATE: 10/2/1   | 2023                    |  |  |  |  |  |  |  |
| Applicant Information  | K.                      |  |  |  |  |  |  |  |
| Applicant Company Name: EASE LLC.  | COV                     |  |  |  |  |  |  |  |
| Contact Person: Tiffany Tonn   | 19                      |  |  |  |  |  |  |  |
| Mailing Address: 1515 FAIRVIEW AVE, STE 205, MISSOULA, MT 5  | 9801                    |  |  |  |  |  |  |  |

"A healthier California where all receive equitable, affordable, and quality health care"

HCAi

STATE OF CALIFORNIA - HEALTH AND HUMAN SERVICES AGENCY

Telephone: (406) 541-3273

Title: Office Assistant

Email: tiffany@easeco.com



## DEPARTMENT OF HEALTH CARE ACCESS AND INFORMATION FACILITIES DEVELOPMENT DIVISION

| Registered Design Professonal Preparing Engineering Recommendations   |  |  |  |  |  |  |  |
|---|--|--|--|--|--|--|--|
| Company Name: EASE LLC  |  |  |  |  |  |  |  |
| Name: Jonathan Roberson California License Number: S4197  |  |  |  |  |  |  |  |
| Mailing Address: 5877 Pine Ave., Suite 210, Chino Hills, CA 91709   |  |  |  |  |  |  |  |
| Telephone: (951) 295-1892   |  |  |  |  |  |  |  |
|   |  |  |  |  |  |  |  |
|   |  |  |  |  |  |  |  |
| HCAI Special Seismic Certification Preapproval (OSP)  |  |  |  |  |  |  |  |
| Special Seismic Certification is preapproved under OSP OSP Number:  |  |  |  |  |  |  |  |
| ORCODE  |  |  |  |  |  |  |  |
| Certification Method  |  |  |  |  |  |  |  |
| Certification Method  |  |  |  |  |  |  |  |
| Testing in accordance with: ICC-ES AC156 FM 1950-16   |  |  |  |  |  |  |  |
| Other(s) (Please Specify):  |  |  |  |  |  |  |  |
| *Use of criteria other than those adopted by the California Building Standards Code, 2022 (CBSC 2022) for component supports and attachments are not permitted. For distribution system, interior partition wall, and suspended ceiling seismic bracings, test criteria other than those adopted in the CBSC 2022 may be used when approved by HCAI prior to testing. |  |  |  |  |  |  |  |
| X Analysis  |  |  |  |  |  |  |  |
| Experience Data  DATE: 10/27/2023   |  |  |  |  |  |  |  |
| Combination of Testing, Analysis, and/or Experience Data (Please Specify):  |  |  |  |  |  |  |  |
| OPVIA DE CODE   |  |  |  |  |  |  |  |
| HCAI Approval   |  |  |  |  |  |  |  |
| Date: 10/27/2023  |  |  |  |  |  |  |  |
| Name: William Staehlin Title: Senior Structural Engineer  |  |  |  |  |  |  |  |
| Condition of Approval (if applicable):  |  |  |  |  |  |  |  |

"A healthier California where all receive equitable, affordable, and quality health care"

**HCA**i

STATE OF CALIFORNIA - HEALTH AND HUMAN SERVICES AGENCY



## **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING**

5877 Pine Ave, Ste. 210 Chino Hills, CA. 91709 Phn: (909) 606-7622

The Department of Health Care Access and Information
PREAPPROVAL OF MANUFACTURER'S CERTIFICATION
OPM-0702

THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE

MANUFACTURER: **werfen** 

**ACL TOP 770 LAS** 

Sheet: 1 of 9

Date: 10/6/23

#### **GENERAL NOTES**

**EQUIPMENT NAME:** 

- 1. THIS HCAI PREAPPROVAL OF MANUFACTURER'S CERTIFICATION (OPM) IS BASED ON THE 2022 CBC. THE DEMANDS (DESIGN FORCES) FOR USE WITH THIS OPM SHALL BE BASED ON THE 2022 CBC
- 2. THIS DOCUMENT MAY ONLY BE USED WITH THE EXPRESS WRITTEN CONSENT OF THE MANUFACTURER LISTED ABOVE FOR THE SPECIFIC PROJECT SITE AND INSTALLATION LOCATION. THIS DOCUMENT IS INVALID WITHOUT SUCH CONSENT.
- 3. THIS PREAPPROVAL CONFORMS TO THE 2022 CALIFORNIA BUILDING CODE WHERE SDS IS NOT GREATER THAN 170, 2.30 & 2.00. SEE DETAIL FOR APPLICABILITY
- 4. FORCES PER ASCE 7-16 SECTION 13.3.1, EQUATIONS 13.3-1, 13.3-2 & 13.3-3.
  - WHERE SDS = 1.70,  $a_p$  = 1.0,  $I_p$  = 1.5,  $R_p$  = 1.5, z/h = 0 AT CONCRETE SLAB. SEE FOLLOWING SHEETS FOR  $\Omega_0$
  - WHERE SDS = 2.30,  $a_D$  = 1.0,  $b_D$  = 1.5,  $b_D$  = 1.0,  $b_D$  = 1.0,
  - WHERE SDS = 2.00,  $a_p$  = 1.0,  $I_p$  = 1.5,  $R_p$  = 1.0, z/h < 1 AT CONCRETE SLAB ON METAL DECK. SEE FOLLOWING SHEETS FOR  $\Omega_0$
- 5. THIS PREAPPROVAL COVERS ONLY THE SUPPORTS AND ATTACHMENTS OF THE EQUIPMENT TO THE STRUCTURE.
- 6. ALL DESIGN FORCES SHOWN ON THE DRAWINGS ARE FACTORED LOADS THAT SHALL BE USED FOR STRENGTH DESIGN.
- 7. CONCRETE SLAB ON METAL DECK DETAIL VALID FOR DEMANDS SHOWN AT ANY ELEVATION IN THE BUILDING. (i.e. z/h < 1)
- 8. CONCRETE SLAB DETAIL VALID FOR DEMANDS SHOWN AT OR BELOW GRADE. (i.e. z/h = 0)

#### 9. RESPONSIBILITIES OF THE STRUCTURAL ENGINEER OF RECORD OF THE BUILDING

- A. PROVIDE SUPPORTING STRUCTURE TO SUPPORT WEIGHTS AND FORCES SHOWN IN ADDITION TO ALL OTHER LOADS.
- B. VERIFY THAT THE INSTALLATION IS IN CONFORMANCE WITH THE 2022 CBC AND WITH THE DETAILS, MATERIAL AND GAGE OF THE UNIT WHERE ATTACHMENTS ARE MADE AGREE WITH THE INFORMATION SHOWN ON THE PREAPPROVAL DOCUMENTS.
- C. VERIFY THAT PROJECT SPECIFIC VALUES OF SDS & z/h RESULT IN SEISMIC FORCES (Eh, Ev ) THAT DO NOT EXCEED THE VALUES ON THE DETAILS.
- D. VERIFY THAT THE CONCRETE SLAB TO WHICH THE EQUIPMENT IS ANCHORED MEETS THI REQUIREMENTS OF THE APPLICABLE ICC ESR REPORT. AND THIS OPM.
- E. VERIFY THAT THE ANCHORS ARE AN ADEQUATE DISTANCE FROM ANY SLAB EDGES OR OPENINGS (SEE TYPICAL DETAIL ON SHEET 2).
- F. VERIFY THAT ALL NEW OR EXISTING ANCHORS ARE AN ADEQUATE DISTANCE FROM THE UNIT ATTACHMENTS AND CHECK FOR INTERACTION WHERE OTHER ANCHORS ARE WITHIN 18" OR 6hef FROM THIS UNIT'S ANCHORS.



#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING**

www.EquipmentAnchorage.com

#### werfen

#### ACL TOP 770 LAS

DES. J. ROBERSON

**JOB NO.** 11-2324

DATE 10/6/23

2 2

9 SHEETS

#### 10. EXPANSION ANCHORS:

A. ATTACHMENT IS TO BE MADE WITH THE ANCHORS LISTED BELOW AND INSTALLED AS DESCRIBED IN THE CORRESPONDING ICC REPORT.


| Anchor Diameter | Concrete<br>Type     | Min. f'c<br>(psi) | Anchor Type         | ICC<br>Report No. | Min.<br>Embed. | Min.<br>Spacing | Min.<br>Edge Dist. | Min. Conc.<br>Thickness | Torque<br>Test | Direct Tension<br>Test |
|-----------------|----------------------|-------------------|---------------------|-------------------|----------------|-----------------|--------------------|-------------------------|----------------|------------------------|
| 1/2"            | Sand Light<br>Weight | 3000              | Hilti Kwik Bolt TZ2 | ESR-4266          | 2.5"           | 7.50"           | 12"                | See<br>Detail "A"       | 50 FT-LB       | N/A                    |
| 1/2"            | Normal<br>Weight     | 3000              | Hilti Kwik Bolt TZ2 | ESR-4266          | 2"             | 3,5"            | 12"                | 4"                      | 50 FT-LB       | 1569 lb                |
| 1/2"            | Normal<br>Weight     | 3000              | Hilti Kwik Bolt TZ2 | ESR-4266          | 2.5"           | 3.5"            | 24"                | 5"                      | 50 FT-LB       | 2031 lb                |

- B. THIS PREAPPROVAL ALLOWS FOR UP TO A MAXIMUM OF 2 ADJACENT CONCRETE SLAB EDGES, 12" & 24" AWAY MINIMUM (i.e. - CORNER).
  SEE ADJACENT DETAIL FOR ADDITIONAL MINIMUM ALLOWABLE CONCRETE
  EDGE DISTANCES.
- C. TESTING AND SPECIAL INSPECTION OF EXPANSION ANCHORS SHALL
  BE PERFORMED BY AN APPROVED INDEPENDENT AGENCY
  EMPLOYED BY THE FACILITY OWNER PER CBC 1704A & 1910A.5


  AND CAC 7-149. ALL REPORTS SHALL BE SENT TO THE INSPECTOR
  OF RECORD, OWNER AND THE ARCHITECT OR ENGINEER IN Itam Staehlin
  RESPONSIBLE CHARGE.
  - (i) AFTER AT LEAST 24 HOURS HAVE ELAPSED SINCE INSTALLATION,
    DIRECT PULL TENSION TEST OR TORQUE TEST AT LEAST 50% OF
    THE ANCHORS.
  - (ii) ACCEPTANCE CRITERIA:
    - DIRECT TENSION TEST: THE ANCHOR SHOULD HAVE NO OBSERVABLE MOVEMENT AT THE TEST LOAD. A PRACTICAL WAY TO DETERMINE OBSERVABLE MOVEMENT IS THAT THE WASHER BECOMES LOOSE.
    - TORQUE TEST: THE APPLICABLE TORQUE MUST BE ACHIEVED WITHIN THE FOLLOWING LIMITS: WEDGE TYPE: 1/2 TURN OF THE NUT
  - (iii) IF ANY ANCHOR FAILS, TEST ALL ANCHORS.
- D. AVOID DAMAGING EXISTING STEEL REINFORCING IN CONCRETE SLAB WHEN INSTALLING CONCRETE EXPANSION ANCHORS.
- E. PROVIDE FOR FULL THREAD ENGAGEMENT OF NUT & WASHER.

#### 1. BOLTS THROUGH CONCRETE ON METAL DECK

- A. BOLTS SHALL BE TORQUED BY 3/4 TURN OF THE NUTS AFTER THE SNUG TIGHT (THE SNUG-TIGHT CONDITION IS DEFINED AS THE TIGHTNESS REQUIRED TO BRING THE CONNECTED PLIES INTO FIRM CONTACT) CONDITION IS ACHIEVED. UNLESS OTHERWISE NOTED.
- B. THROUGH BOLT HOLES SHALL BE 1/16" LARGER THAN BOLT SIZE (HOLE SIZE = BOLT SIZE + 1/16) FOR CONCRETE.
- C. THROUGH-BOLTS IN CONCRETE SHALL RECEIVE SPECIAL INSPECTION AND TESTING (THROUGH BOLTS WITH STEEL TO STEEL CONNECTION IN TENSION DO NOT REQUIRE TENSION TESTING) IN ACCORDANCE WITH REQUIREMENTS FOR POST-INSTALLED ANCHORS.

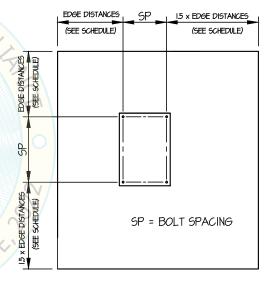

TYPICAL CONCRETE EDGE DETAIL

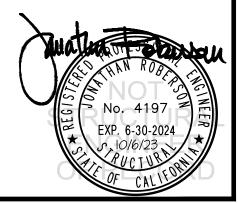

#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING**

www.EquipmentAnchorage.com

#### werfen

#### ACL TOP 770 LAS

DES. J. ROBERSON

11-2324 JOB NO.


10/6/23 DATE

SHEET

SHEETS

#### SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB

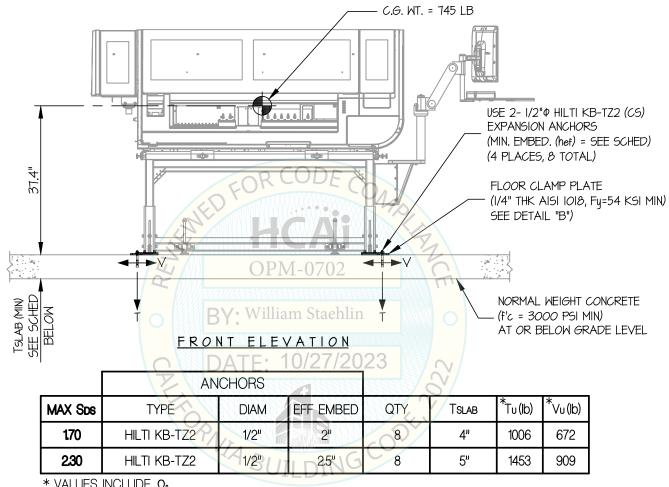

#### NOTES:

- \* VALUES INCLUDE Ω<sub>0</sub>
- FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: 2p = 1.0, 1p = 1.5, 2p =
- 2. THIS PREAPPROVAL ENCOMPASSES WEIGHTS AND VERTICAL C.G. POSITIONS NOT EXCEEDING VALUES SHOWN.
- 3. THIS PREAPPROVAL WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION, COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED. BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR). USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS. AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- SEE GENERAL NOTES: SHEETS 1 AND 2



# EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING WWW.EquipmentAnchorage.com SHEET

werfen

ACL TOP 770 LAS

DES. J. ROBERSON

**JOB NO.** 11-2324

DATE 10/6/23

1


9 SHEETS

OF

SEISMIC SUPPORTS & ATTACHMENTS

MAX Sps ≤ 1.70

CONCRETE SLAB

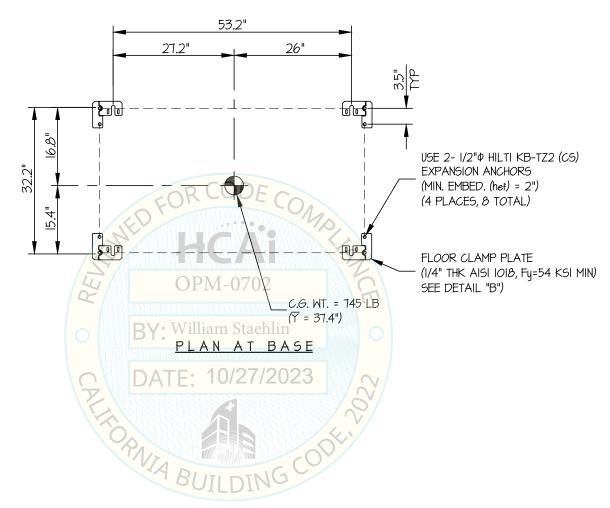



#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING** www.EquipmentAnchorage.com SHEET

werfen

DES. J. ROBERSON

OF

ACL TOP 770 LAS

11-2324 JOB NO. 10/6/23


DATE

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

MAX Sps < 2.30

CONCRETE SLAB

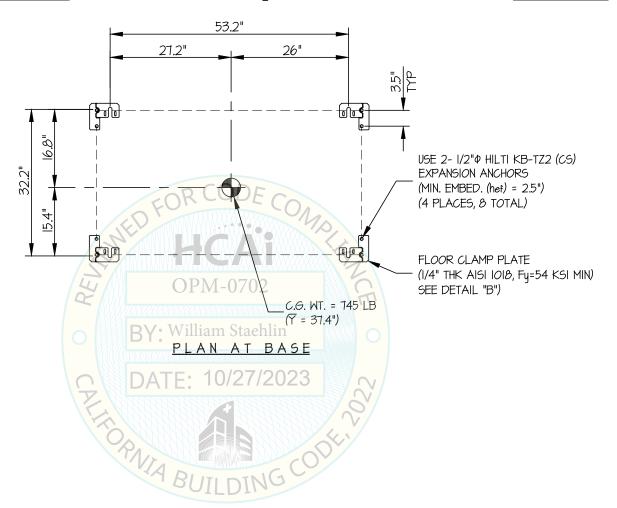



#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING**

www.EquipmentAnchorage.com

#### werfen

#### DES. J. ROBERSON

SHEET

#### ACL TOP 770 LAS

10/6/23 DATE


JOB NO.

11-2324

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

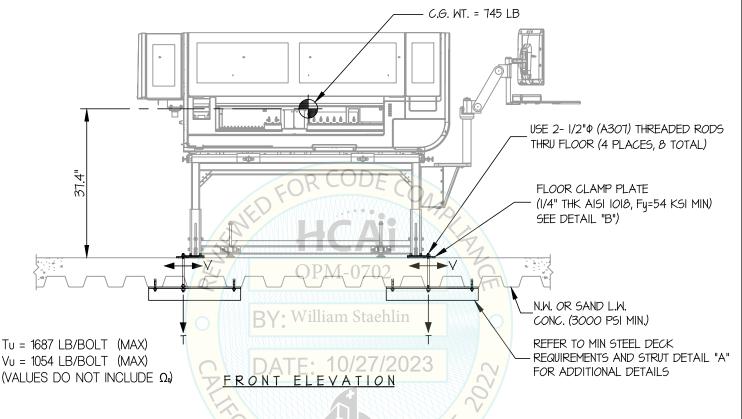

NOTES:

1. FORCES ARE DETERMINED PER 2022 CALIFORNIA BUILDING CODE AND ASCE 7-16. STRENGTH DESIGN IS USED. (EXAMPLE: SDS = 2.00, 2p = 1.0, p = 1.5, p = 1

> HORIZONTAL FORCE (En) = 2.40 Wp

HORIZONTAL FORCE (Emh) = 4.80 Wp (FOR CONCRETE ANCHORAGE)

VERTICAL FORCE (Ev) = 0.40 Wp

- 2. THIS PREAPPROVAL ENCOMPASSES WEIGHTS AND VERTICAL CG POSITIONS UP TO THE VALUES SHOWN.
- 3. THIS PREAPPROVAL WAS PREPARED WITHOUT KNOWLEDGE OF ANY SITE CONDITION. COMPATIBILITY FOR USE WITH A SITE SHALL BE EVALUATED BY THE STRUCTURAL ENGINEER OF RECORD OF THE INSTALLATION (SEOR). USE REQUIRES APPROVAL BY THE SEOR.
- 4. STRUCTURAL ENGINEER OF RECORD FOR THE INSTALLATION SHALL VERIFY ALL CONDITIONS, EVALUATE INTERACTION WITH ADJACENT EQUIPMENT AND ANCHORS, AND PROVIDE SUPPORT STRUCTURE DESIGNED TO SUPPORT WEIGHTS AND FORCES SHOWN IN COMBINATION WITH ALL OTHER LOADS THAT MAY BE PRESENT.
- 5. SEE GENERAL NOTES; SHEETS 1 AND 2.



#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING** www.EquipmentAnchorage.com DES. J. ROBERSON werfen

ACL TOP 770 LAS


11-2324 JOB NO. 10/6/23 DATE

SHEET

SHEETS

SEISMIC SUPPORTS & ATTACHMENTS

CONCRETE SLAB ON METAL DECK

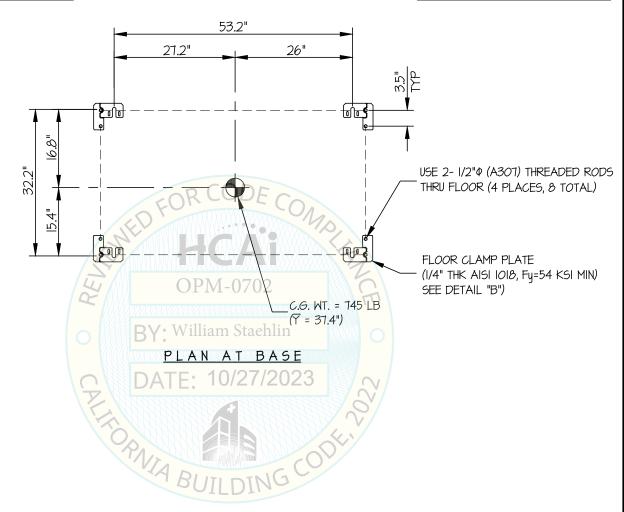



# EASE

#### **EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING**

www.EquipmentAnchorage.com

werfen

DES. J. ROBERSON


JOB NO.

11-2324


8HEET

ACL TOP 770 LAS

DATE 10/6/23 OF

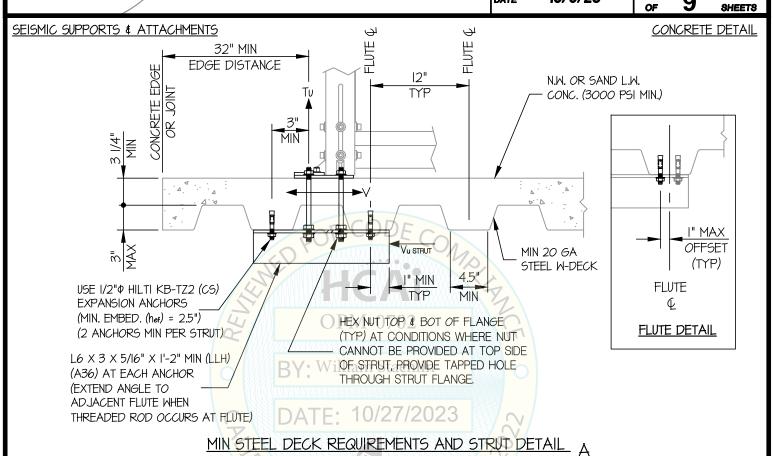

PVIA BUILDING CODY

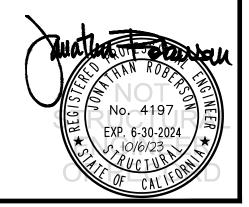

# EQUIPMENT ANCHORAGE & SEISMIC ENGINEERING www.EquipmentAnchorage.com

werfen

ACL TOP 770 LAS

DES. J. ROBERSON

**JOB NO.** 11-2324


DATE 10/6/23

9

OF

BRACKET DETAIL

SEISMIC SUPPORTS & ATTACHMENTS

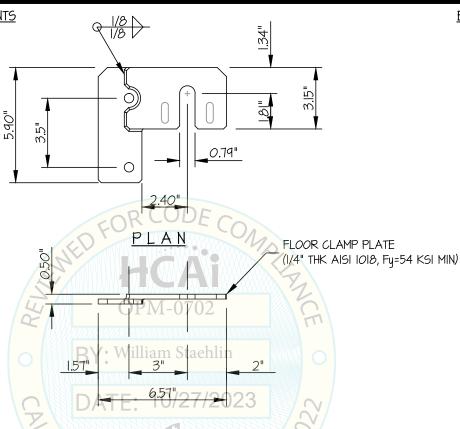

BRACKET DETAIL (B)

FRONT

