

Nurse Call System Testing and Inspection Checklist

Test Date:	Room #:	
Normal Call Placed from Patient Pillow Speaker(s)	Bed-1	Bed-2
Alerts Nurses' Console w/Distinct Tone		
Console Displays Room# and Bed#		
Audio Between Pillow Speaker and Nurses' Console		
Activates Room's Dome w/Distinct Light		
Activates Zone w/Distinct Light		
Must be Cancelled from Room		
Pillow-speaker Disconnect Alarm at Console		
Normal Call Placed From Push-Button on Cord(s)	Bed-1	Bed-2
Alerts Nurses' Console w/Distinct Tone		
Console Displays Room# and Bed#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome w/Distinct Light		
Activates Zone w/Distinct Light		
Must be Cancelled from Room		
Pillow-speaker Disconnect Alarm at Console		
Bathroom Pull-Switch(es)	Toilet	Shower
Alerts Nurses' Console w/Distinct Tone		
Console Displays Room#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome w/Distinct Light		
Activates Zone with Distinct Light		
Alerts Switchboard Console		
Staff-Assist Call from Bed Station	Bed-1	Bed-2
Alerts Nurses' Console with Distinct Tone		
Console Displays Room#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome with Distinct Light		
Activates Zone with Distinct Light		

Alerts Switchboard Console		
Must be Cancelled from Room		
Staff Emergency from Bed Station	Bed-1	Bed-2
Alerts Nurses' Console with Distinct Tone		
Console Displays Room#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome with Distinct Light		
Activates Zone with Distinct Light		
Alerts Switchboard Console		
Must be Cancelled from Room		
Code-Blue Button on Bed Station	Bed-1	Bed-2
Alerts Nurses' Console with Distinct Tone		
Console Displays Room#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome with Distinct Flashing Light		
Activates Zone Light with Distinct Flashing Light		
Alerts Switchboard Console		
Must be Cancelled from Room		
Smoke Detector Interface	Bed-1	Bed-2
Alerts Nurses' Console with Distinct Tone		
Console Displays Room#		
Audio Between Bed Station and Nurses' Console		
Activates Room's Dome with Distinct Flashing Light		
Activates Zone with Distinct Flashing Light		
Alerts Switchboard Console		
Must be Cancelled from Room		
Staff Locating Device		
Nurses' Console Displays Room # of Staff badge		
Rooms Dome Light Activates on Staff Presence		